




# Safer Patient Positioning



New  
Product

# Pink: The New Green

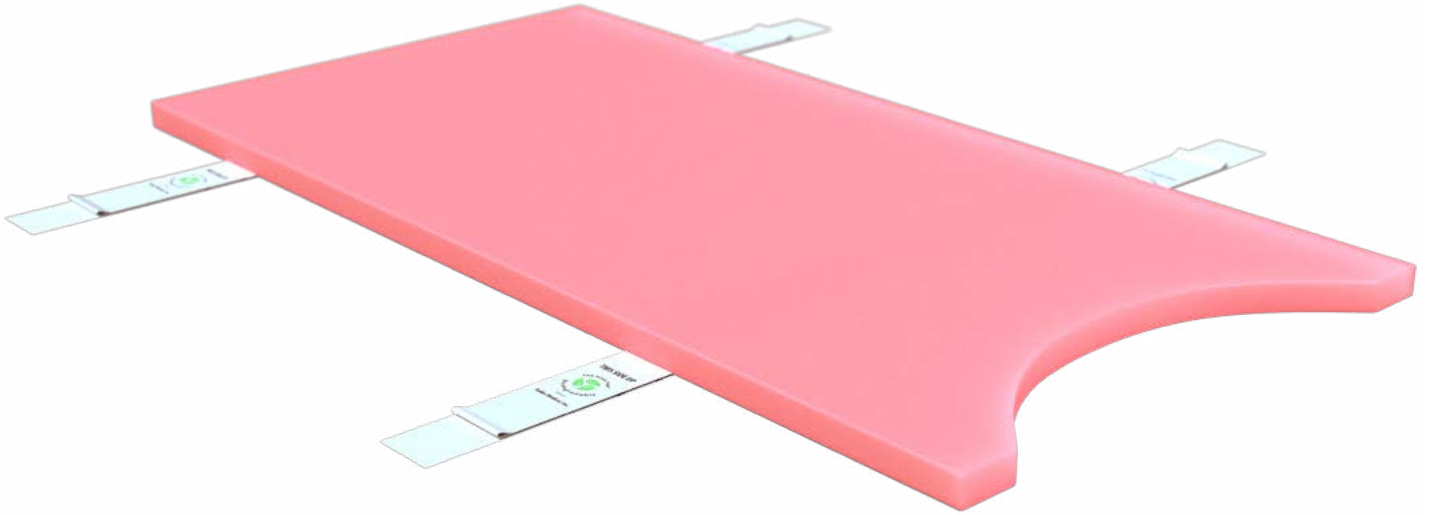

## The Pink Pad<sup>®</sup>. Now **Biodegradable**.

The safest Trendelenburg patient positioning system is now biodegradable; another first from Xodus Medical. With nearly a dozen US Patents and counting, Pink Pad solutions from Xodus Medical are simply first in patient (and environmental!) safety.

Independent Laboratory Tested & Certified to ASTM D5511 Standards for Biodegradability

Our Most Popular Kit #40695




Xodus Medical proudly supports the [AORN Tool Kit](#) for the Prevention of Perioperative Pressure Injury. Scan to view up-to-date patient safety recommendations or visit [xodusmedical.com/AORNtoolkit](http://xodusmedical.com/AORNtoolkit)

# The Pink Pad<sup>®</sup>

The Safest, Most Clinically-Proven and Preferred Trendelenburg Positioning Solution

Patent No. 8,464,720; 8,511,314; 9,161,876; 9,750,656; 9,782,287; 9,931,262; 9,949,883; 10,045,902; 10,285,890; 10,324,050  
2,859,398; 11,484,456

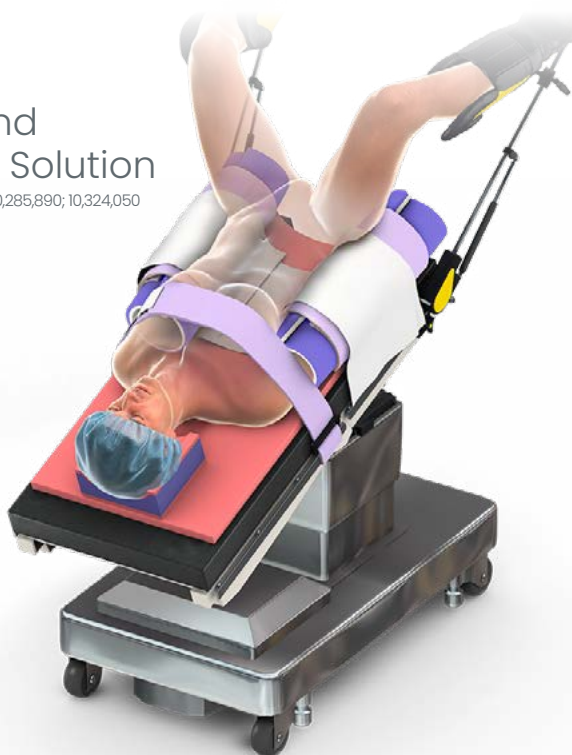

TRENDELENBURG

## Maximize Safety in Trendelenburg

The Pink Pad Trendelenburg Positioning System is an innovative, patented, clinically-preferred solution for increasing patient safety in Trendelenburg—preventing unwanted patient movement, while protecting nerves and skin against Hospital Acquired Pressure Injuries (HAPIs).



### Safely Prevents Patient Sliding

Overcome the positioning challenges of steep Trendelenburg; even with heavy, high-BMI patients.



### Utmost Protection Against HAPIs

Patented, immersive, enveloping materials are breathable, soft, moisture-wicking and insulative.



### Proven Pressure Reduction

In a recent study, subjects on the Pink Pad ranging from 16–54 BMI (35.5 BMI avg.) saw a **43%** peak-pressure reduction.



### Significant Time Savings

Setup time under one minute could save your OR up to \$1,900 per procedure (value of OR time: \$100/minute)



## A Material Difference

Clinical leaders in the world's top hospitals trust the Pink Pad's unique, HAPI-reducing material to maximize patient safety in steep Trendelenburg








New  
Product

# The Pink Pad® AIRCLOUD ASSIST

Safer, Easier Patient Transfer and Trendelenburg Positioning

Integrating The Pink Pad® into both top & bottom surfaces ensures skin protection, as well as patient stability in both Trendelenburg & Reverse Trendelenburg

**TRENDELENBURG**  
& Reverse Trendelenburg

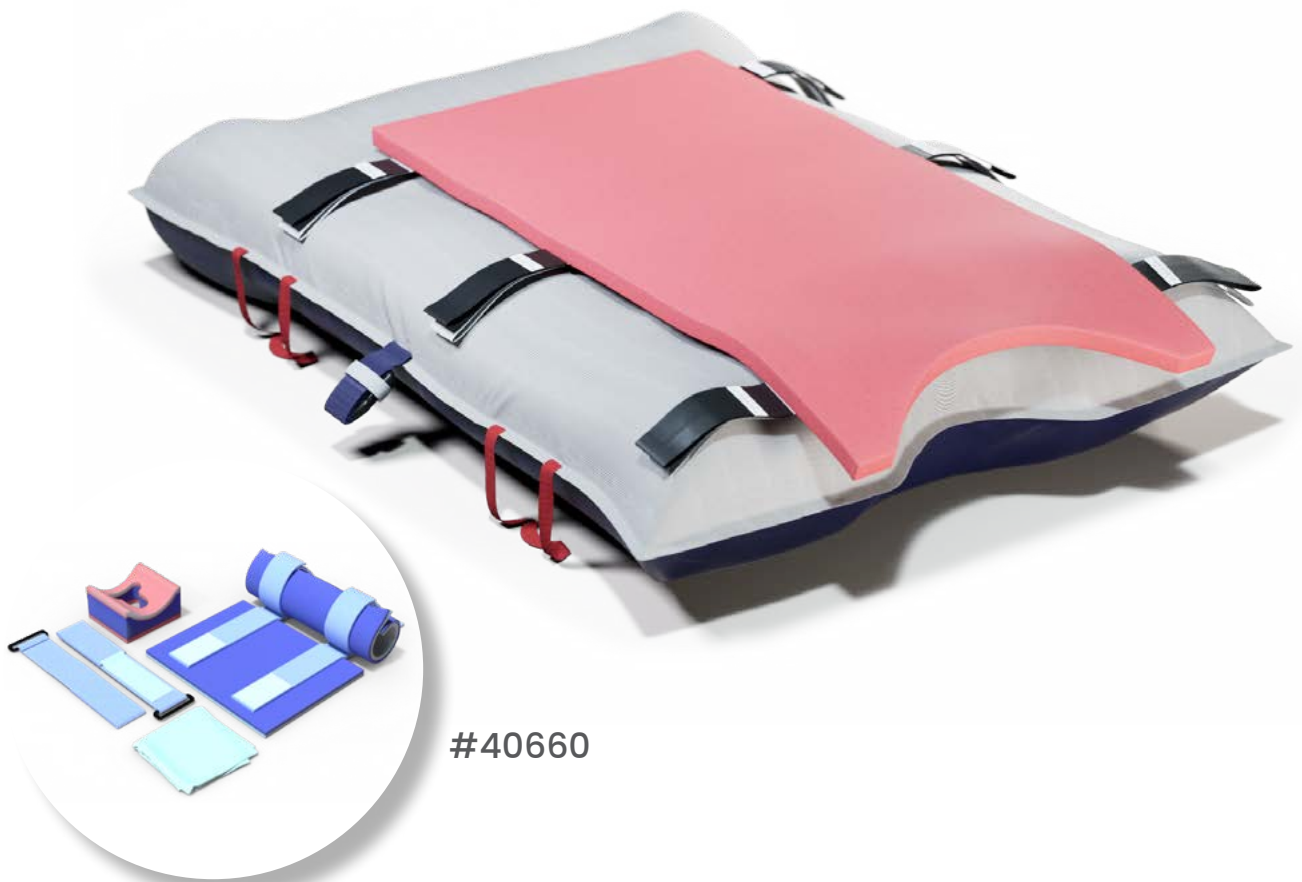

#40660

- Lift-free transfer and repositioning for patients of all sizes
- Safely position high-BMI patients while protecting against Hospital Acquired Pressure Injuries and nerve damage
- Establish a clinical evidence-based best practice for reducing peak pressure and preventing unwanted patient movement



# The **Pink Pad** – One Step

A Safe, Streamlined, and Complete Trendelenburg Positioning System

- Establish a best practice standard-of-care<sup>1</sup> with a single-step, streamlined patient positioning system
- Set up time under one minute saves your hospital up to \$1,900 per procedure\*

\*Based on OR costs of \$100/minute<sup>2,3</sup>, and 20-minute average set up times for competitive systems



1: Journal of Gynecologic Surgery, 2021; A Prospective Randomized Trial of Antislip Surfaces During Minimally Invasive Gynecologic Surgery


2: OR Management News: What Is a Minute Worth in the OR? June 2018

3: Critical Care Medicine: January 2019 - Volume 47 - Issue 1 - p 41

## The **Pink Pad** – Safe Assist

The Safest, Most Clinically-Proven and Preferred Trendelenburg Positioning Solution – Now Easier to Use

- Easily transfer and position patients with integrated straps
- Safely prevent patient sliding, HAPI formation, and nerve damage in Trendelenburg positions
- Establish a clinical evidence-based best practice for reducing peak pressure and preventing unwanted patient movement

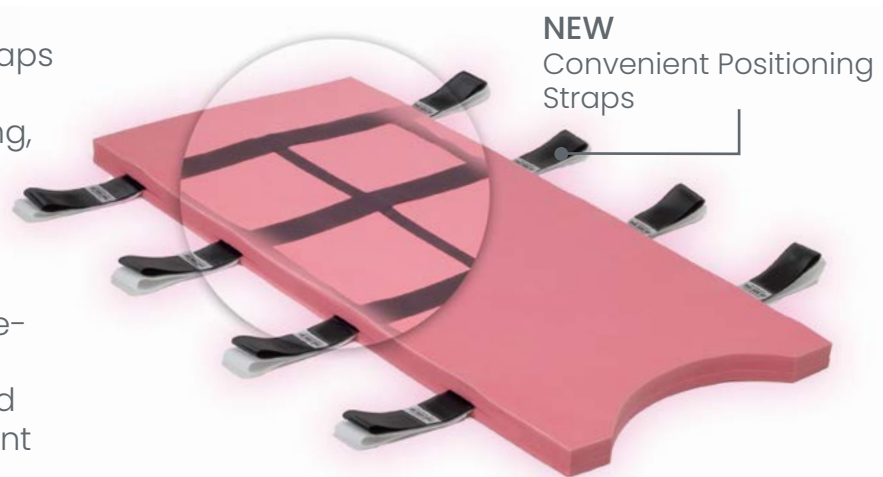

# The **Pink** Reverse-T Kit™

U.S. Patents: 8,464,720; 8,511,314; 9,161,876; 9,750,656; 9,782,287; 9,931,262; 9,949,883; 10,045,902; 10,285,890; 10,324,050; 11,484,456 Additional Patents Pending

REVERSE TRENDELENBURG



Optional One-Step™  
Arm Protectors  
Available (#40433)

## Safely Prevent Sliding

An innovative seat contour made of high-density, supportive material ensures anatomically correct positioning, while skin-friendly PinkProtect™ safely prevents unwanted movement in Reverse-T.

## Reduce Risk of HAPIs

Unique, patented materials provide complete protection against HAPIs; minimizing non-reimbursable care costs by avoiding preventable injuries and CMS Never-Events.


## Complete Coverage for Every Patient

The Reverse-T Kit requires less than one minute to set up, while a single-use design minimizes cross-contamination risk and improves OR turnover times.



Reverse-T Footboard  
Pad (20"x12"x2")  
(#40548)



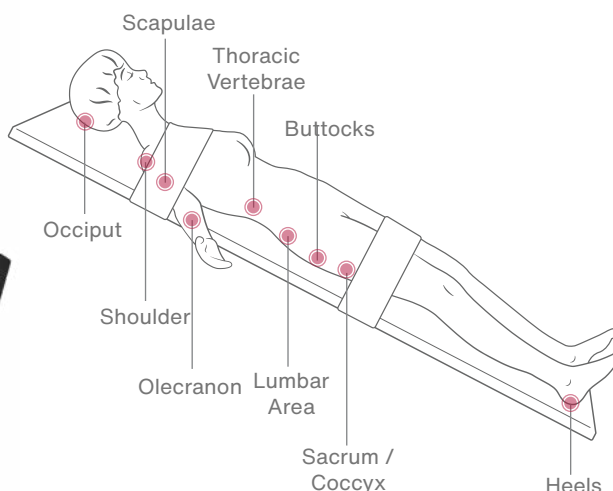

With patients weighing 450lbs or more, and BMIs in excess of 65, safe positioning can be a real challenge. Xodus Medical's Pink Reverse-T Kit not only prevented our patients from sliding, but did so safely.

— Alicia C, RN, BSN, Surgical Services Manager, Hartford, KY



## Popular Configuration (#40640) Includes:

- 1 Pink Pad Reverse Trendelenburg Pad w/ Posterior Support Contour
- Large Armboards (Pair) - with DermaProx™
- 1 Lift Sheet
- 2 Sets of Skin-Friendly Body Straps
- Single-Use Adult Head Positioner
- #40548 Reverse-T Footboard Pad (20"x12"x2")



## Popular Configuration (#40643) Includes:

- 1 Pink Pad XL
- Large Armboards (Pair) - with DermaProx™
- 1 Lift Sheet
- 2 Sets of Skin-Friendly Body Straps
- Single-Use Adult Head Positioner
- #40548 Reverse-T Footboard Pad (20"x12"x2")





# The Pink Hip Kit™

## Postless Positioning System

Patent No. 8,464,720; 8,511,314; 9,161,876; 9,750,656; 9,782,287; 9,931,262; 9,949,883; 10,045,902; 10,285,890; 10,324,050; 11,484,456

## HIP PROCEDURES



#72205289

Exclusively  
Distributed  
by Smith+Nephew



### Eliminate Nerve Injury

Safely and effectively position patients without the need for a perineal post



### Protection from Head to Toe

Maximize traction as well as protection against HAPIs with Hip Arthroscopy Boot liners



#72205682

## Patient Safety Advantages

- Significantly reduces risk of HAPI formation
- Postless approach requires shorter traction time, makes distraction easier, and improves C-arm visualization to allow for a more complete dynamic exam
- Significantly reduces post-operative pain and recovery times
- Requires no change to surgical technique
- Allows for a greater range of motion during surgery, making complex procedures easier
- Kits available for various tables

"The nice thing with a postless system is that it truly prevents skin and nerve damage to the perineum. Hip Arthroscopy typically takes from 1-2 hours to perform, and a large amount of pressure is applied to the perineum area. That pressure can be associated with skin injury and nerve injury in as many as 30-35% of cases. If you can eliminate the post, you can actually reduce that complication rate to 0%.

— Joshua D. Harris, MD, Associate Professor of Ortho Surgery, Houston, TX



Xodus Medical proudly supports the **AORN Tool Kit** for the Prevention of Perioperative Pressure Injury. Scan to view up-to-date patient safety recommendations or visit [xodusmedical.com/AORNtoolkit](http://xodusmedical.com/AORNtoolkit)



# PinkProtectKits™

## Pressure Injury Prevention for Prolonged Patient Positioning

Patent No. 8,464,720; 8,511,314; 9,161,876; 9,750,656; 9,782,287; 9,931,262; 9,949,883; 10,045,902; 10,285,890; 10,324,050; 11,484,456



#40784



#40521



#40519

Multiple Head Positioner  
Options Available



#40545

#40524

#40544

Multiple Pink Arc  
Options Available

PROLONGED

- Featuring PinkProtect™ – an immersive, enveloping, breathable and moisture-wicking material for the utmost protection against Hospital Acquired Pressure Injuries (HAPIs)
- Multiple materials with varying densities to deliver optimal pressure distribution
- Proven, skin-friendly positioners designed to improve patient safety throughout prolonged surgical procedures
- Unique designs tailored to the specific requirements of OR patient positioning

More Options Available at [xodusmedical.com/PinkProtectOR](http://xodusmedical.com/PinkProtectOR)



#40499  
**PinkProtect™**  
Elevation Wedge



#40600  
**Pink Pad EXT™**



#40547  
**Dual Layer Armboard  
Pads w/ Straps**




# The Pink Lat Pack™

A New Standard for Safe, Effective, and Efficient Lateral Positioning

Patent No. 8,464,720; 8,511,314; 9,161,876; 9,750,656; 9,782,287; 9,931,262; 9,949,883; 10,045,902; 10,285,890; 10,324,050; 11,484,456

## Adjustable Lateral Head Positioner

Skin-friendly materials, adjustable height, and an ergonomic design make this the ideal lateral head positioner

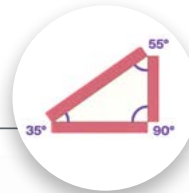

## Multi-Angle Wedge

35°, 55°, and 90° inclination options optimize positioning for every patient and procedure




#LPK10005



## Safety Maximized

Increase patient safety with DermaProx™ and PinkProtect™: breathable, moisture-wicking materials that gently immerse and envelop areas prone to pressure injury.

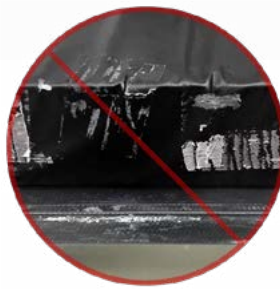

## Standard of Care Emphasized

No customization required. Regardless of weight or BMI; The Pink Lat Pack™ provides a safe, consistent standard of care for every patient, in every procedure, every time.



## Efficiency Optimized

In clinical trials, an average of 15-20 minutes were saved during setup. When necessary, The Pink Lat Pack™ makes repositioning faster, safer, and easier.


LATERAL DECUBITUS



"Prior to working with Xodus Medical, a solution for lateral positioning that was both safe and efficient simply did not exist. The Pink Lat Pack™ combines skin-friendly materials with smart configurations that ensure a consistent standard of care. They save our OR a significant amount of time, especially during setup. Most importantly, the safety of our patients is better than ever."

— Dr. Domenico Savatta, MD  
Chief of Minimally Invasive & Robotic Adult Urologic Surgery  
New Jersey


## Pink Lat Pack™ Available Configurations

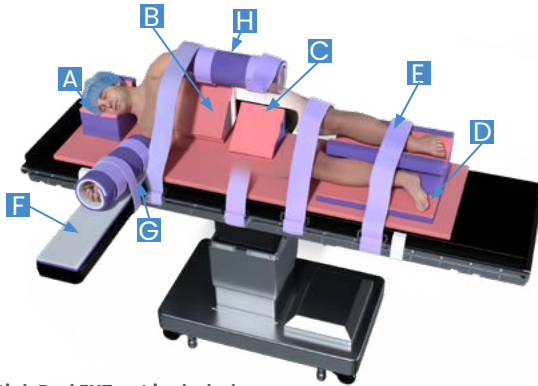

\*Pink Pad EXT not included  
\*Not all components are pictured

### #LPK10004 – Savatta Pink Lat Pack™


- A. Adjustable Lateral Head Positioner (1)
- B. Multi-Angle Body Wedge (1)
- C. Small Body Wedge (1)
- D. Lateral Leg Positioner (Savatta Style) (1)
- E. Skin-Friendly Body Straps (7)
- F. Armboard Pads with DermaProx™ (1pr)
- G. Skin-Friendly Armboard Straps (1pr)
- H. One-Step™ Arm Protectors with DermaProx™ (1pr)

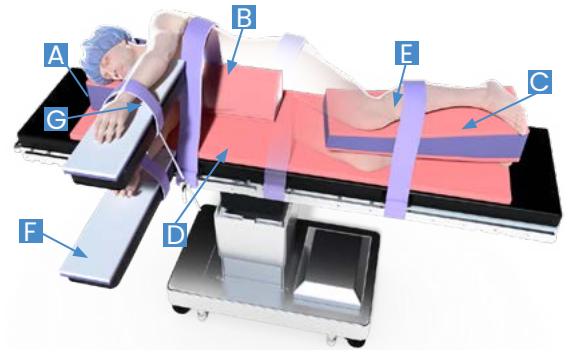

\*Not all components are pictured

### #LPK10005 – Standard Pink Lat Pack™


- A. Adjustable Lateral Head Positioner (1)
- B. Multi-Angle Body Wedge (1)
- C. Tapered Intra-Leg Positioner (1)
- D. Pink Pad EXT™ (1)
- E. Skin-Friendly Body Straps (4)
- F. Armboard Pads with DermaProx™ (1pr)
- G. Skin-Friendly Armboard Straps (1pr)
- H. Lift Sheet (1)

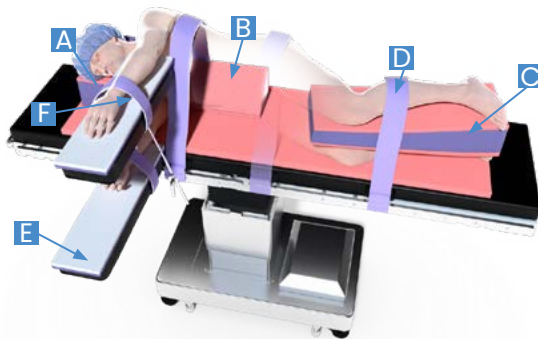

\*Pink Pad EXT not included  
\*Not all components are pictured

### #LPK10006 – Modified Pink Lat Pack™

- A. Adjustable Lateral Head Positioner (1)
- B. Multi-Angle Body Wedge (1)
- C. Tapered Intra-Leg Positioner (1)
- D. Skin-Friendly Body Straps (4)
- E. Armboard Pads with DermaProx™ (1pr)
- F. Skin-Friendly Armboard Straps (1pr)

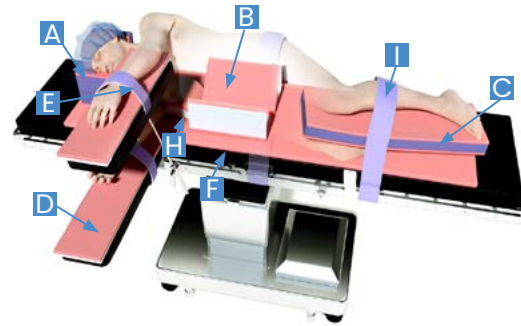

\*Not all components are pictured

### #LPK10015 – Pink Lat Pack™: Thoracic

- A. Adjustable Lateral Head Positioner (1)
- B. Low Profile Lateral Wedge (2)
- C. Intra-Leg Positioner (1)
- D. Armboard Pads with PinkProtect™ (1pr)
- E. Skin-Friendly Arm Board Straps (2)
- F. Pink Pad EXT™ (1)
- G. Lift Sheet (1)
- H. Pink Arc (Axillary Roll) (1)
- I. Skin-Friendly Body Straps (3)

LATERAL DECUBITUS



# The **Pink** Peg Board Kit™


## LATERAL DECUBITUS




- A new standard for safe, effective and efficient lateral positioning.
- PinkProtect™ materials are immersive, enveloping, soft, breathable, and help maximize positioning security.
- Help prevent Hospital Acquired Pressure Injuries (HAPIs).
- Ultra-supportive cores distribute pressure over the most challenging positioning surfaces.



Unique high-density Peg Covers feature a seamless design that optimally balances HAPI reducing characteristics, as well as patient support and pressure distribution.

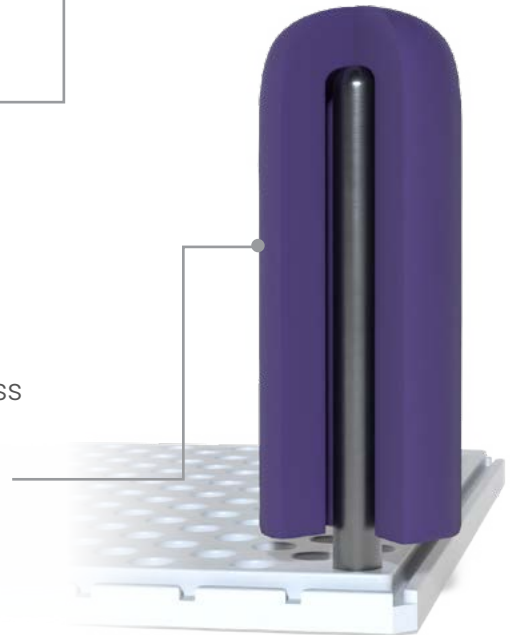



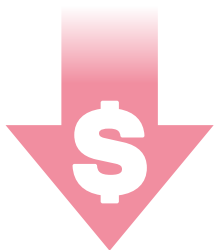

Reduce set up time, cost,  
and eliminate cross-  
contamination risks.



Setup time: less than  
one minute.


REPEATABLE  
**BEST**  
PRACTICE

Eliminate the need  
for custom, off-label  
positioning solutions.

LATERAL DECUBITUS

### Our Most Popular Configuration (PBK10001) Includes:

- PinkProtect™ Armboard Pads w/ Straps
- PinkProtect™ Peg Board Pad
- Peg Board Adjustable Lateral Head Positioner
- Universal High-Density Peg Covers (4)
- Skin-Friendly Body Strap and Lift Sheet

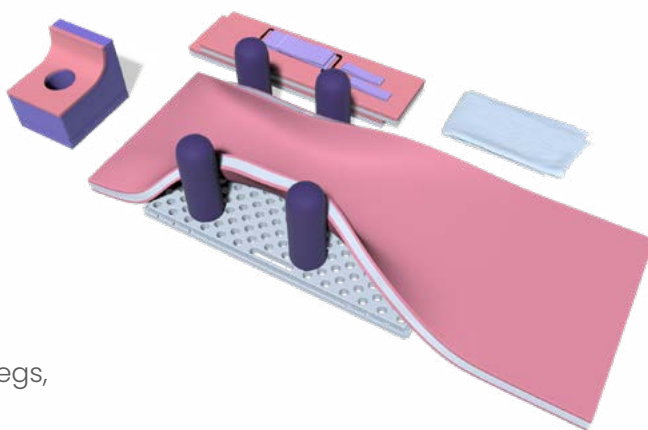

### Helpful Accessories Sold Separately:

- **#40945** 3" Intra-Leg Positioner (shown at right)
- Pink Dual Layer Pads (additional protection for legs, feet, etc)
  - **#40771** (15" x 11" x 1")
  - **#40772** (8" x 11" x 1")
- PinkProtect Pink Arcs (Axillary Rolls)
  - **#40544** PinkProtect Standard Pink Arc (9" x 8" x 4")
  - **#40524** PinkProtect Medium Pink Arc (16" x 6" x 4")
- **#40549** Universal High-Density Peg Covers (case qty 16)
- **#40567** High-Density Double Peg Covers (case qty 16)



#40567:  
Double Peg  
Covers


### Additional Lateral Solutions

## The **Pink** Stulberg Kit

- Help prevent Hospital Acquired Pressure Injuries (HAPIs)
- Achieve a precise fit for your specific equipment



#40778

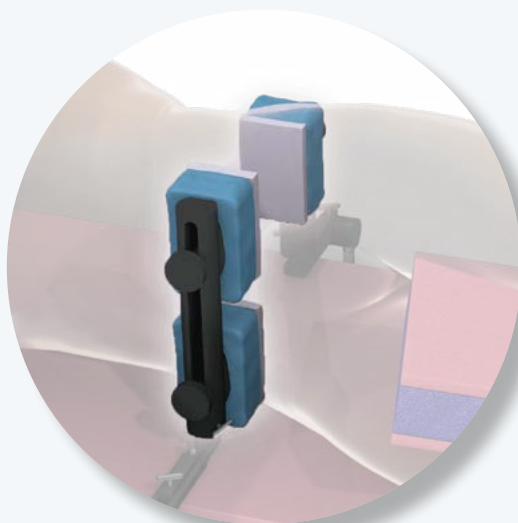


Visit [xodusmedical.com/Stulberg](http://xodusmedical.com/Stulberg)  
for Kit Options and to Learn More

# Surgical BackPacks™

Safe Spinal Table Positioning

PRONE & SPINAL

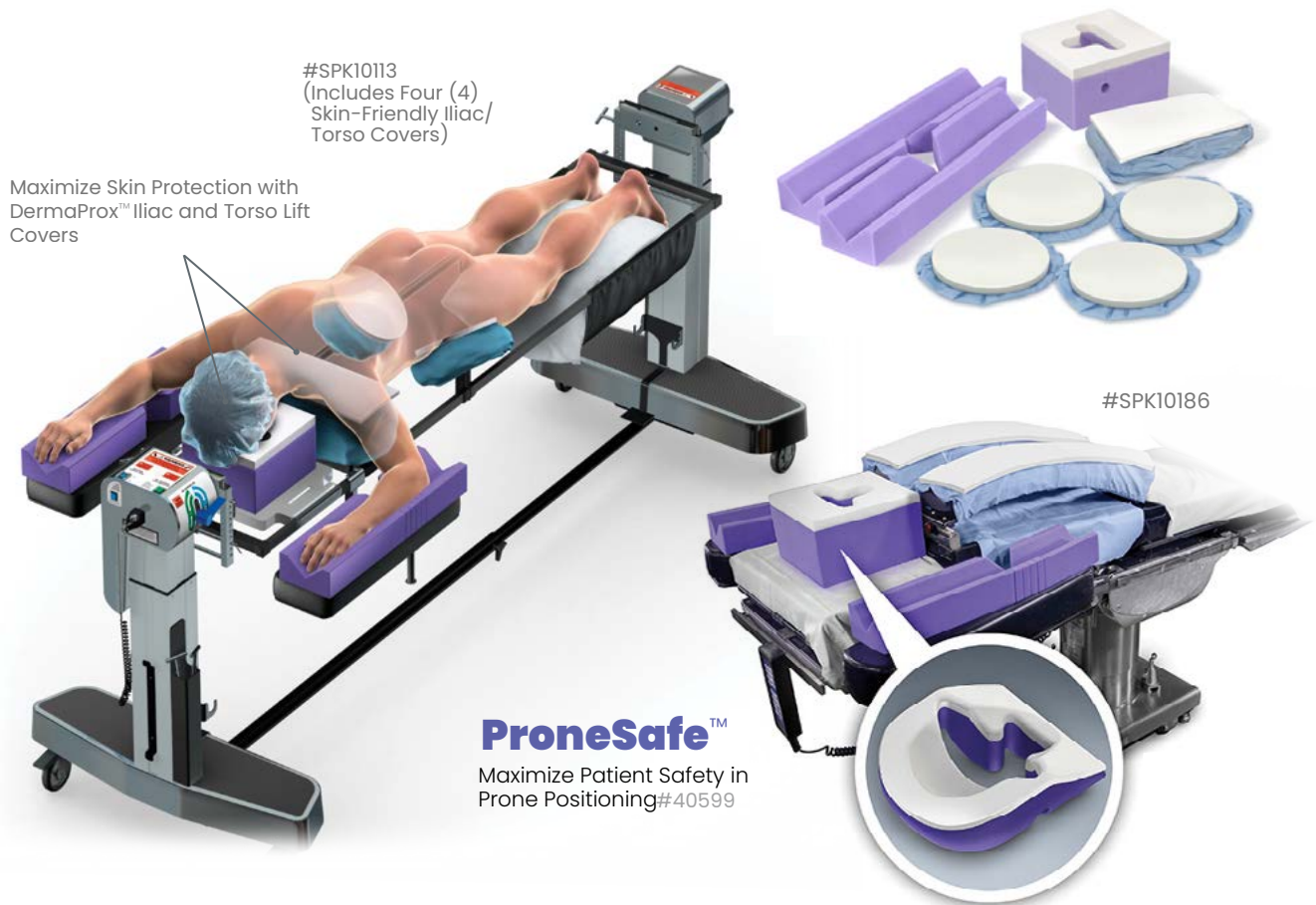

## Patient Safety Advantages

- Improve HAPI protection with DermaProx™; a unique, skin-friendly material that provides a soft, breathable, immersive, enveloping and moisture-wicking interface between the patient and the surgical table
- Enhance pressure distribution on high-risk patient areas and bony prominences
- Maximize patient safety while minimizing non-reimbursed care costs; avoid preventable injuries and CMS Never-Events
- Achieve a precise fit for your specific equipment with customizable kits

"The Xodus products have greatly decreased skin redness and breakdown on the chin and bony prominences during long spinal cases. I have greater peace of mind using these products knowing that our patients are safe and better protected."


— Mary Ann Pruskowski, RN, BSN, BSHA, CNOR, Neuro



# Shoulder Surgery Positioning Solutions

Complete Solutions for Proper, Safe Patient Positioning on a Multitude of Surgical Tables and Chairs

SHOULDER

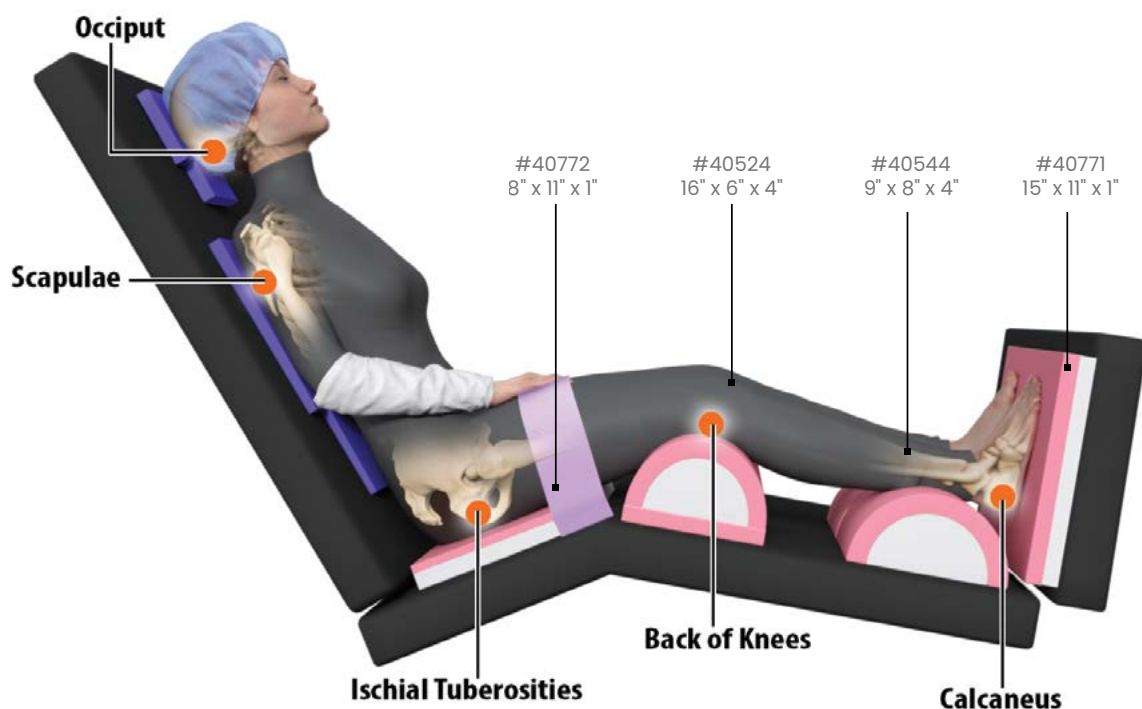

Enhanced Patient Safety For Your Preferred Positioning Technique



#40763  
Beach Chair Head  
Restraint with  
DermaProx™



#40762  
Two-Piece Beach  
Chair Head Wrap  
with DermaProx™



#90403  
Shoulder Arthroscopy  
Kit



# Anesthesia Positioning Solutions

Provide anatomical support and complete protection against Hospital Acquired Pressure Injuries (HAPIs), while optimizing airway management

ANESTHESIA



**40445**  
Prone Head Positioner with  
PinkProtect™ Top




**40521**  
Prone Head  
Positioner with  
PinkProtect™  
Top & Bottom



**40770**  
Side-Prone Head  
Positioner  
with PinkProtect™ Top



**40471**  
Lateral Head Positioner  
with Adjustable  
Heights & PinkProtect™  
Top

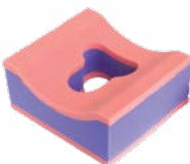

**40519**  
Cradle Head  
Positioner  
with PinkProtect™  
Top



**40541**  
Low Profile Cradle  
Head Positioner with  
PinkProtect™ Top



**NS30430**  
Nasotracheal  
Tube  
Immobilizer



**40518**  
Supine Head Positioner



**40466**  
Ocular Head Positioner  
with Disposable Cover



**40436**  
Vitrectomy Head Positioner



**40599**  
ProneSafe Head Positioner  
(Large)

**40699** (Pediatric)



**40448**  
7" Prone Head Positioner  
with DermaProx™ Top

**40468** (Pediatric)






# Maggio Adjustable Prone Head Positioner

New  
Product

Safer Skin Protection and Spinal Alignment for Surgical Patients

PRONE

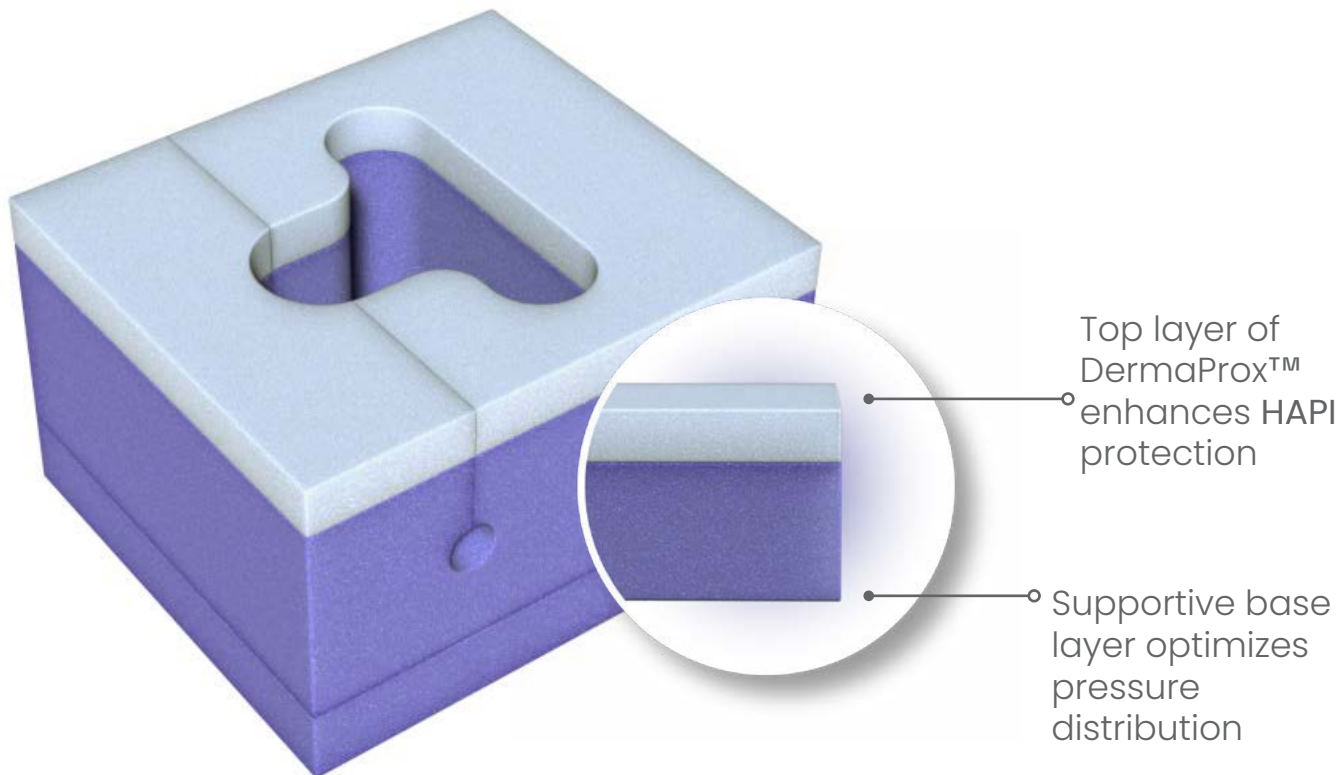

- Various height settings adapt to the individual needs of your patients
- Multiple layers enhance patient safety
  - Skin-friendly DermaProx™ top layer protects against Hospital Acquired Pressure Injuries (HAPIs)
  - Materials optimized for both skin protection and long-term support
- Open-cell material helps manage moisture; protect patients' skin



# Safer Single-Use Positioning

#40404


## Slotted Head Positioner

- Provides anatomically correct positioning of the head and neck

#40409

## Abduction Pillow

- Promotes proper positioning for optimal healing after surgery
- Concave sides and skin-friendly straps allow for a safe and secure fit

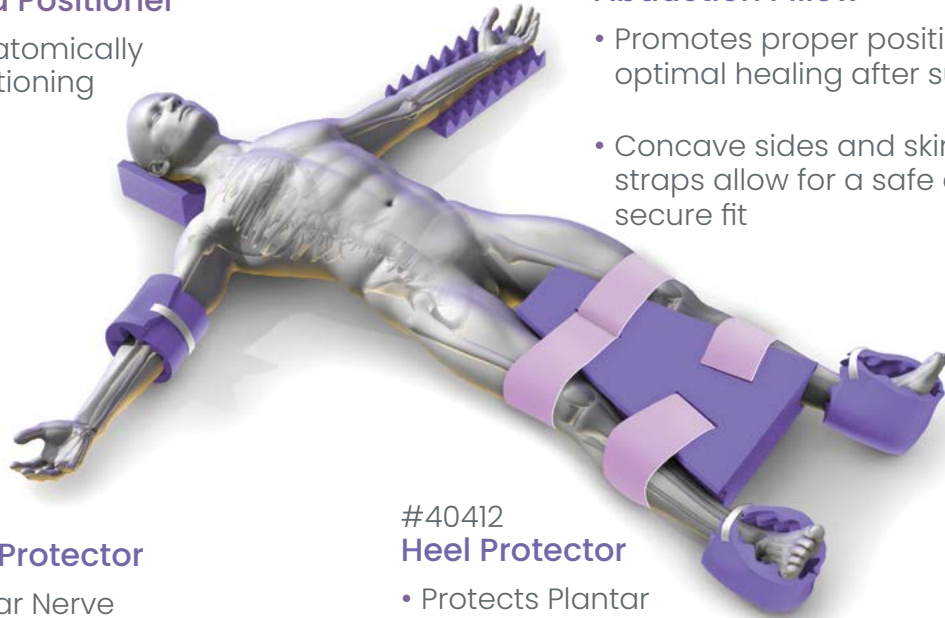

#40411

## Ulnar Nerve Protector

- Protects Ulnar Nerve from pressure related complications

#40412

## Heel Protector

- Protects Plantar Nerve

## Essential Positioning



**40432**  
Deluxe  
Laminectomy  
Arm Cradle



**40420**  
Standard  
Laminectomy  
Arm Cradle



**40411**  
Ulnar Nerve  
Protetor



**40412**  
Heel Protector

## Straps



**30210**  
Arm Board Strap



**30220**  
Knee and Body Strap

## Orthopedic Positioning



**40410**  
Abduction  
Pillow, Large



**40409**  
Abduction  
Pillow, Medium




**40408**  
Abduction  
Pillow, Small



# DERMAPROX™ PADS

## A Necessity for Every Pressure Injury Prevention Program

- Meets recommendations set forth by the AORN Tool Kit for the Prevention of Perioperative Pressure Injury
- Breathable, moisture-wicking, pressure-distributing material helps prevent HAPI occurrence
- Protects skin without the use of sticky adhesives; the removal of which may cause skin tears

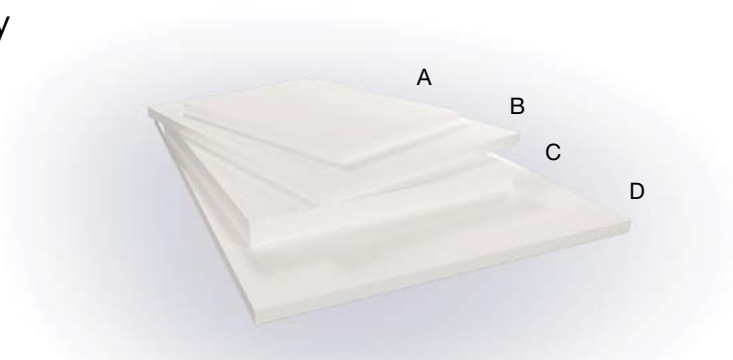

Product #	Dimensions
(A) 40527	11" x 8" x 0.5"
(B) 40528	11" x 15" x 0.5"
(C) 40529	11" x 15" x 1"
(D) 40514	22" x 14" x 0.5"

DERMAPROX

## Bariatric Positioning



**40497**  
Abduction Pillow, Small



**40452**  
Prone Head Positioner  
w/ DermaProx™ Top



**40454**  
Donut Head Positioner  
w/ DermaProx™ Top



**40437**  
Laminectomy Arm  
Cradle



**40486**  
Child Prone Head  
Positioner  
w/ Dermaprox™ Top  
(small)



**40451**  
Chest Roll



**40449**  
Body Wedge



# Improving Perioperative Outcomes Through Safe Patient Positioning



Continuing Education Provider



Funding Provided by



## Contact Xodus Medical to Schedule a Free CE Session!



Xodus Medical proudly supports the [AORN Tool Kit](#) for the Prevention of Perioperative Pressure Injury. Scan to view up-to-date patient safety recommendations or visit [xodusmedical.com/AORNtoolkit](http://xodusmedical.com/AORNtoolkit)



**XODUS MEDICAL**  
Making Surgery Safer™

702 Prominence Drive  
New Kensington, PA 15068,  
United States

[xodusmedical.com](http://xodusmedical.com)  
[info@xodusmedical.com](mailto:info@xodusmedical.com)  
Tel: 1+724-337-5500

FDA REGISTERED  
ISO 13485 CERTIFIED



MADE IN THE  
**USA**



Follow Us on  
**Social Media**