


**SINGLE USE KNEE AND BODY STRAP INSTRUCTIONS FOR USE**

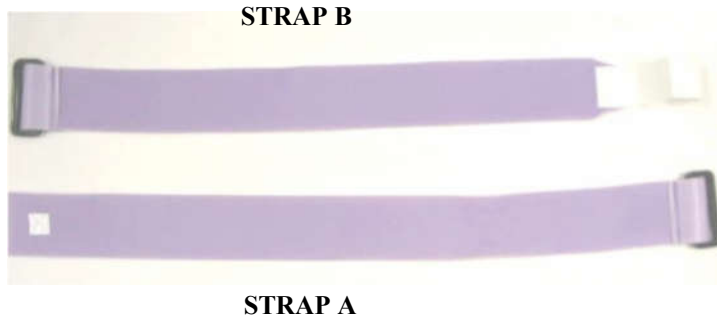


**ATTACHING STRAPS TO BED/TABLE:**

**ATTACHING STRAPS TO EACH OTHER**




**STEP 1**  
Slide end of strap A around bed/table frame and through end clasp. Make sure Hook square is face up, smooth side should

always face patient.

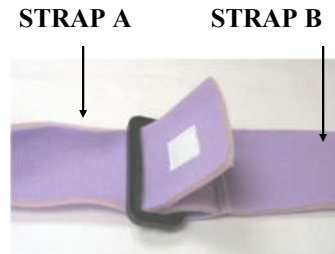

**STEP 1**  
Place end of strap A through the clasp at the end of strap B and fasten. Smooth side should face patient.

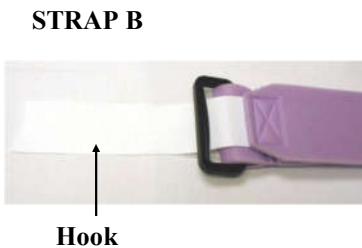

**STEP 2**  
Repeat above step with strap B hook side face down around the other side of the bed/table frame and through the end clasp.




**STEP 2**  
Place strap around the table/bed and up around the patient. Slide the end of strap B through the plastic clasp of strap A and across to fasten.



**STEP 3**  
Belts will look like this when properly attached to the bed/table frame.



**STEP 3**  
This is how the straps should look when fastened across the patient's legs.

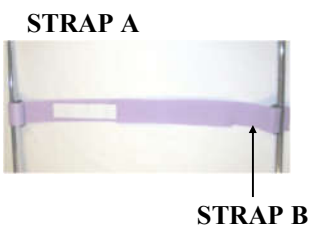

**STEP 4**  
Place the end Of strap B with the hook across strap A.