






# INSTRUCTIONS FOR USE: BOOT LINER - ARTHREX



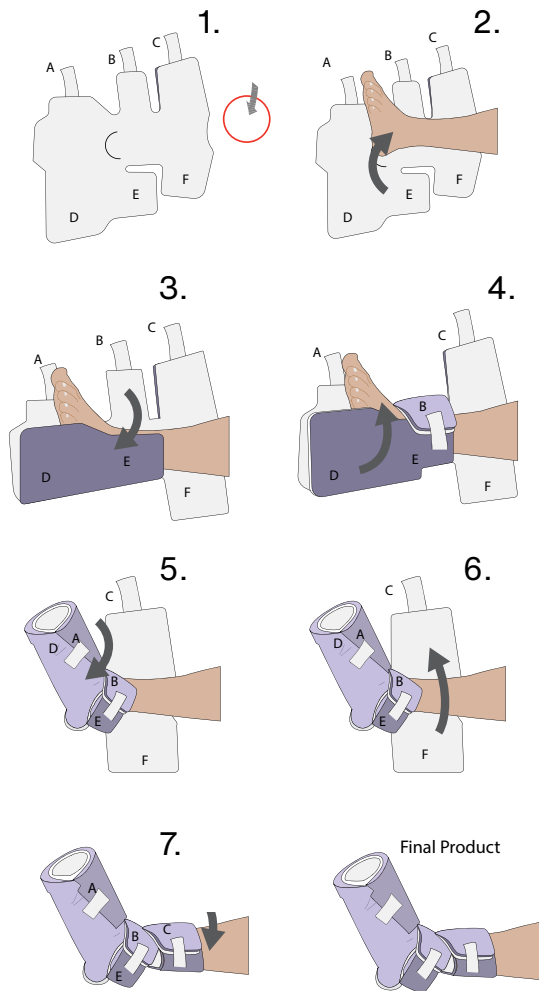



1. Unfold bootliner and have the white DermaProx™ layer facing upwards.
2. Place patient's heel in the middle of the pad so that the notch aligns with patient's calf, and the cutout aligns with patient's heel.
3. Wrap non-hook, part D & E, around patients foot.
4. Place strap B (hook & loop side) of middle strap on top of part E so the patient's ankle is wrapped. Be sure not to over-tighten.
5. Wrap bottom half of pad around the rest of the patients toes and place strap A onto part D so that the top of the patient's foot is wrapped. Be sure not to overtighten..
6. Wrap part F around the rest of the patients calf.
7. Place strap C on top of part F so that the patient's calf is wrapped. Be sure not to overtighten.

## Symbol Glossary

	Medical Device
	Manufacturer
	Batch Code
	Quantity

	Do not use if package is damaged
	Do not re-use
	No Latex
	Authorized representative in the European Community

	Consult electronic instructions for use
	Non-Sterile
	European Conformity
	Catalog number



702 Prominence Drive | Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555

xodusmedical.com  
info@xodusmedical.com

FDA REGISTERED  
ISO 13485 CERTIFIED

LITIFUAT002 R0 6/2/2022