

GREENGUARD CORNER PROTECTORS

INSTRUCTIONS FOR USE

BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

DESCRIPTION

The GreenGuard Corner Protectors are a reusable product made of a soft, pliable material for use on the bottom, sides, and/or top of surgical instrument trays to provide protection of sterilization wrap during sterilization, transportation, and/or storage.

PRIOR TO USE

Inspect the GreenGuard Corner Protectors for any damage such as cuts, nicks, or deformities.

INDICATIONS

Disinfection and Sterilization shall be in accordance with each facility's internal protocol for sterilization of similar use devices, such as Steam Sterilizers, Surgical Instrument Washers, etc.

INTENDED USE

The GreenGuard Corner Protectors are to be used on the bottom, sides, and/or top of surgical instrument trays to provide protection of sterilization wrap during sterilization, transportation, and/or storage.

CONTRAINDICATIONS

This device is not designed, sold, or intended for use except as indicated.

CLINICAL BENEFITS

To aid hospital facilities in protection of the instrument tray sterilization wrap in order to maintain sterility and need for reprocessing when damaged.

PRECAUTION

- Use this product only for its intended purposes, as described in this manual.
- **Notice to the User and/or patient that any serious incident that has occurred in relation to the device should be reported and the competent authority of the Member State in which the user and/or patient is established, as well as Xodus Medical.**

WARNINGS

Follow hospital protocol for Disinfection and Sterilization shall be in accordance with each facility's internal protocol for sterilization of similar use devices, such as Steam Sterilizers, Surgical Instrument Washers, etc.

DISPOSAL

GreenGuard Corner Protectors are a reusable device. Dispose of the device if any damage occurs from use or handling.

TECHNICAL SPECIFICATION


Materials of manufacture include: Thermoplastic Elastomer (TPE)

STORAGE, TRANSPORT, AND OPERATIONAL CONDITIONS

The GreenGuard Corner Protectors should be stored in a clean, dry location at room temperature prior to use.

PREPARATION AND USE

1. GreenGuard Corner Protectors: apply Corner Protectors to any and all tray corners in need of protection (typically both the tray's bottom four (4) corners, as well as the top four (4) corners).
2. GreenGuard Straight-Edge Protectors: apply Straight-Edge Protectors to any and all tray sides in need of protection.
3. Once Corner and Edge Protectors are in place, wrap the tray according to hospital protocols.
4. GreenGuard Corner Protectors are compatible with both Steam and Sterrad Sterilization processes.
5. Continued Use:
 - a. GreenGuard Corner protectors are designed to be reusable
 - b. Cleaning of Green Guard per hospital protocol
 - c. GreenGuard Corner Protectors can be recycled
6. Discontinue use if Corner Protectors are visibly damaged, worn, or degraded.

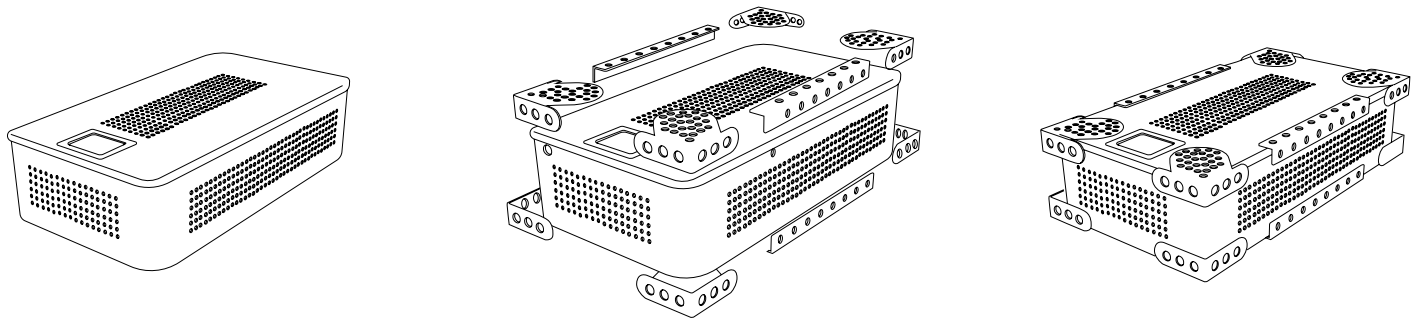

Caution, Consult
Accompanying
Documents



xodusmedical.com/eifu

Consult instructions
for use or consult
electronic
instructions for use



Not Made with
Natural Rubber
Latex



Do Not Use If The
Package Is
Damaged Or
Opened



XODUS MEDICAL
Making Surgery Safer™

702 Prominence Drive
New Kensington, PA 15068,
United States

xodusmedical.com
info@xodusmedical.com
Tel: 1+724-337-5500

FDA REGISTERED
ISO 13485 CERTIFIED



MADE IN THE
USA

Patents Pending