





DermaProx™

Quick Usage Guide

Supine Position

- Use a DermaProx sheet under the following bony prominences to redistribute pressure: heels, shoulders, sacrum
- Use DermaProx sheets on top of pillows or gel rolls under the knees and heels to make reusable supports more skin friendly and reduce unwanted movement
- Use DermaProx sheets to pad along the patient's arms to protect the ulnar and radial nerves
- Tear or cut DermaProx sheets to pad along IV and other line sites
- Pad "sleds" to protect the patient's arm
- Place DermaProx over gel head rests to redistribute pressure across back of the head

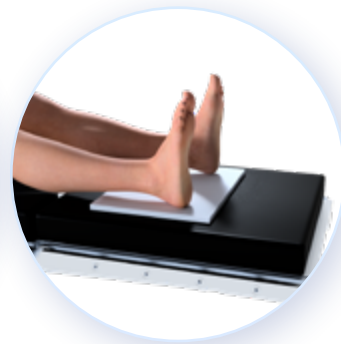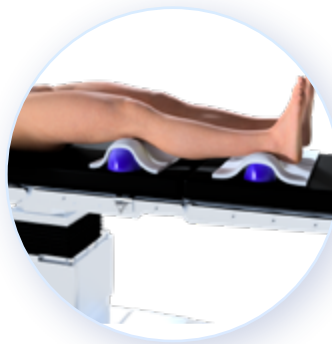

Lithotomy Position

- Use a DermaProx sheet under the following bony prominences to redistribute pressure: heels, shoulders, sacrum
- Use DermaProx sheets to pad along the patient's arms to protect the ulnar and radial nerves
- Tear or cut DermaProx sheets to pad along IV and other line sites
- Pad "sleds" to protect the patient's arm
- Place DermaProx over gel head rests to redistribute pressure across back of the head
- Use DermaProx sheets to pad stirrup liners to reduce unwanted movement and pressure injury risk





Lateral Position

- Use DermaProx sheets over bean bags to pad and redistribute pressure along spine, hip, abdomen
- Use DermaProx sheet over gel or other reusable devices used as an axillary roll
- Use DermaProx sheets on top of pillows placed between the legs to protect the skin and reduce unwanted movement
- Use DermaProx sheets to pad the lateral arm rest or over pillows placed between the arms to protect the ulnar/radial nerves
- Tear or cut DermaProx sheets to pad along IV and other lines
- Place DermaProx sheets under dependent hip to reduce pressure
- Place DermaProx sheets under reusable safety straps (or tape) to protect skin
- Use on head positioners to reduce unwanted movement and protect nerves/skin of face/ear



Prone Position

- Use DermaProx sheets to pad between and over breasts to reduce unwanted movement and pressure
- Use DermaProx sheets to pad hips and thighs to reduce unwanted movement and pressure
- Use DermaProx sheets under knees to redistribute pressure
- Use DermaProx sheets on top of pillows and other reusable devices to support lower legs
- Use DermaProx sheets to pad arms on armboards or when tucked at sides
- Tear or cut DermaProx sheets to pad along IV and other lines
- Use DermaProx sheets to pad OR table rails where necessary

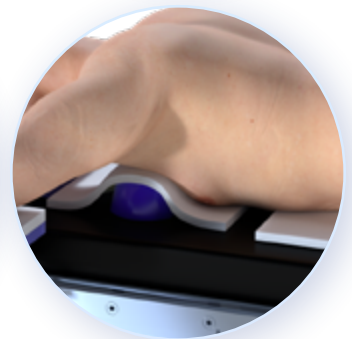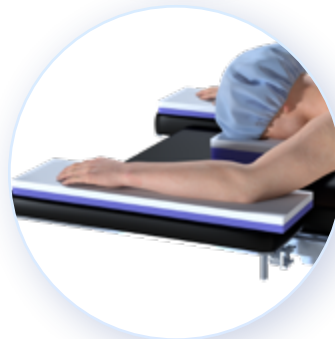


Fowler's/Beach Chair

- Use DermaProx sheets to pad pillows and other reusable devices under knees, lower calf, and feet
- Use DermaProx sheets to pad the ischial tuberosities (sit bones)
- Use DermaProx sheets to pad the shoulders
- Use DermaProx sheets to pad the arms and IV as well as other lines
- Use DermaProx sheets to pad the foot board if applicable



Reverse Trendelenburg

- Use DermaProx sheets to pad pillows and other reusable devices under knees and feet
- Use DermaProx sheets to pad the sacrum
- Use DermaProx sheets to pad the shoulders
- Use DermaProx sheets to pad the arms and IV as well as other lines
- Use DermaProx sheets to pad the foot board if applicable

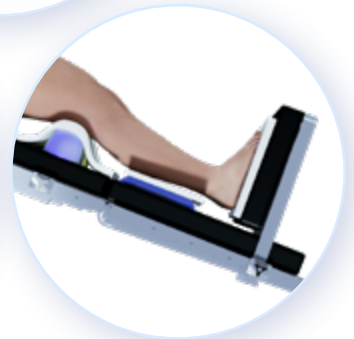

XODUS MEDICAL
Making Surgery Safer™

702 Prominence Drive
New Kensington, PA 15068,
United States

xodusmedical.com
info@xodusmedical.com
Tel: 1+724-337-5500

FDA REGISTERED
ISO 13485 CERTIFIED



MADE IN THE
USA



Follow Us on
Social Media



Scan for a
Free Sample

DermaProx™ Pads

INSTRUCTIONS FOR USE

BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

DESCRIPTION

DermaProx™ Pads are non-sterile, single-use devices designed to provide a breathable barrier between the patient's skin and hard surfaces. Constructed of moisture-wicking, adhesive-free material, these pads promote airflow to keep the skin dry and intact while reducing the risk of abrasion, shear, blistering, or tearing.

They are suitable for use in both operating rooms and critical patient care areas, including ICUs, and may be layered or folded to meet various positioning and protection needs.

INDICATIONS

DermaProx™ Pads are intended for use as protective skin barriers during surgical procedures and in critical care settings. They are designed to:

- Protect skin from contact with hard equipment such as IV lines, catheters, and monitoring leads.
- Provide cushioning and reduce pressure on bony prominences.
- Serve as an alternative to blankets or egg-crate foam for skin protection.
- Promote breathability and moisture control at the skin surface.

INTENDED PURPOSE AND USER

This device is intended for use by trained healthcare professionals only.

CONTRADICTIONS

This device is not designed, sold, or intended for use except as indicated.

KNOWLEDGE AND USE


Professional use requires knowledge of this instruction for use. Device use limited to surgical operating room in a hospital or surgery center.

CLINICAL BENEFITS

- Promotes skin breathability and moisture management.
- Reduces the risk of skin redness, blistering, and tearing.
- Provides protective cushioning against abrasion and shearing.
- Eliminates risks associated with adhesives, such as compromised airflow or skin residue.

PREPARATION AND USE

- Open the package and remove the DermaProx™ Pads.
- Position pads over or under the patient area or equipment to be covered and protected.
- Pads may be folded or layered to provide additional support or protection.
- Ensure pads are securely placed to maintain coverage and patient comfort.
- Continuously monitor the patient's skin integrity and circulation throughout use.

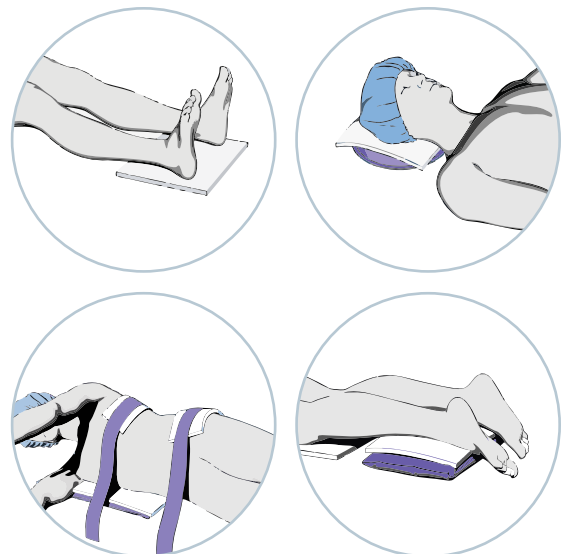

- Follow your facility's policies and protocols for patient monitoring.
- Safeguard the device from exposure to prep solutions.

DISPOSAL

- After use, dispose of DermaProx™ Pads in accordance with hospital policy.

WARNINGS

- Single-use only. Reuse or reprocessing may result in device failure, contamination, or patient injury.
- Do not reuse or reprocess this device.
- DermaProx™ Pads should not be used in direct contact with open wounds.
- After use, the DermaProx™ Pads should be disposed of in accordance with hospital policy.
- After use, dispose of the pads according to hospital policy.
- Monitor patient skin for integrity and circulation during use.

PRECAUTIONS

- DermaProx™ Pads are intended for use in perioperative and critical care settings only.
- Use this device in accordance with the instructions for use and with attention to any contraindications, warnings, or precautions.
- Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.
- Notice to the User and/or patient that any serious incident that has occurred in relation to the device should be reported and the competent authority of the Member State in which the user and/or patient is established, as well as, Xodus Medical and its Authorized Representative.

TECHNICAL SPECIFICATIONS

- Materials of manufacture include:
 - » Polyurethane foam
- Shelf Life – indefinite

STORAGE, TRANSPORT, AND OPERATIONAL CONDITIONS

- Store DermaProx™ Pads in a clean, dry location at room temperature prior to use. Avoid prolonged exposure to elevated temperatures.



Medical Device



Non-Sterile



xodusmedical.com/eifu

Consult Electronic
Instructions for Use



Do Not
Re-use



Not Made with
Natural Rubber
Latex



Do Not Use If
The Package Is
Damaged Or
Opened



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland



XODUS MEDICAL
Making Surgery Safer™

702 Prominence Drive
New Kensington, PA 15068,
United States

xodusmedical.com
info@xodusmedical.com
Tel: 1+724-337-5500

FDA REGISTERED
ISO 13485 CERTIFIED



MADE IN THE
USA

LITIFUDP003 R1 10/17/2025