

INSTRUCTIONS FOR USE

EMESIS CONTAINMENT POUCH

DESCRIPTION:

Emesis bags are disposable, non-sterile, single-use devices designed to hold vomitus. Each bag is equipped with a secure closure mechanism to prevent spillage and contamination. They are made of leak-proof, durable materials and are easy to handle.

INDICATIONS

Emesis bags are intended for use by healthcare professionals or patients to capture and dispose of vomitus in a sanitary and controlled manner.

INTENDED USE:

Emesis bags are designed to collect vomitus from patients in medical environments, including hospitals, clinics, and emergency settings. They provide a hygienic and safe way to manage vomiting, reducing the risk of contamination and infection.

CONTRAINDICATIONS

This device is not designed, sold, or intended for use except as indicated.

KNOWLEDGE AND INTENDED SETTING

Device intended for use by trained healthcare personnel. Device is intended to be used in a surgical setting, and operating room in a hospital or surgery center. They are used in settings such as emergency rooms, patient wards, post-operative care, chemotherapy units, and ambulances.

CLINICAL BENEFITS:

- **Reduction of Cross-Contamination:** By providing a leak-proof and easy-to-dispose container, emesis bags help reduce the risk of spreading infectious diseases.
- **Enhanced Infection Control:** The disposable nature of the emesis bags minimizes the need for cleaning reusable containers, thus improving infection control measures.

INTENDED USER

This device is intended for use by trained healthcare professionals only.

INTENDED CARE SETTINGS

Hospitals and Ambulatory Surgery Centers (ASC).

WARNINGS

- For single-use only. Do not reuse.
- Use only for the intended purpose of collecting vomitus.

PRECAUTIONS

- Ensure that the bag is properly sealed after use to prevent leakage.
- Periodically check the dispenser for signs of wear or damage and replace if necessary.
- Do not use the bag if it shows any signs of damage or leakage.
- Dispose of used emesis bags in accordance with hospital protocol.

PREPARATION AND USE

A. Preparing the Emesis Bag:



1. Open the packaging and remove the emesis bag.
2. Ensure that the bag is fully expanded and that the closure mechanism is functioning properly.

B. Using the Emesis Bag:

1. Position the bag near the patient's mouth to catch vomitus.
2. Once the patient has finished vomiting, seal the bag using the closure mechanism.
3. Dispose of the sealed emesis bag in accordance with the healthcare facility's biohazard disposal protocols.

C. Post-Use:

1. Remove gloves and other protective equipment in line with hospital protocols.
2. Wash hands thoroughly after handling the emesis bag.

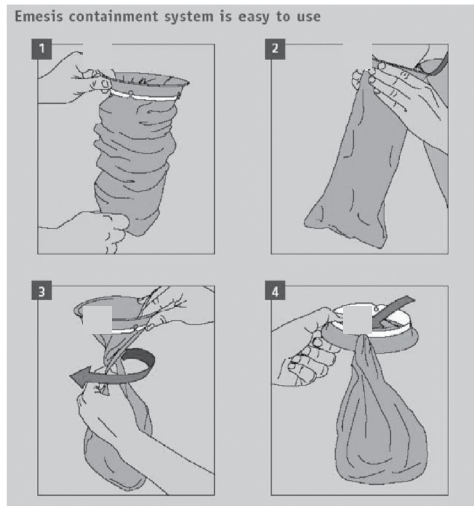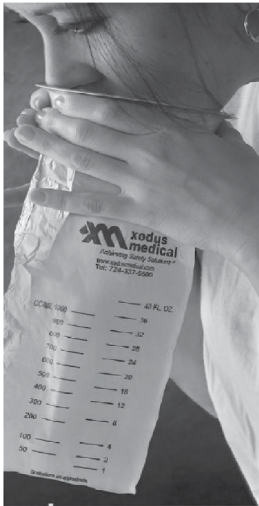

DISPOSAL

Dispose of the emesis bags in accordance with hospital protocol.

TECHNICAL SPECIFICATIONS

- Emesis Bag: Polyethylene
- Emesis Bag Ring: Polyethylene

STORAGE, TRANSPORT, AND OPERATIONAL CONDITIONS

- The emesis bags should be stored in a clean, dry location at room temperature prior to use.
- Avoid exposure to extreme temperatures or humidity.



xodusmedical.com/eifu

Consult instructions
for use or consult
electronic
instructions for use



Non-sterile



Not Made with Natural
Rubber Latex



Do Not Re-use



Do Not Use If The
Package Is Damaged
Or Opened



XODUS MEDICAL
Making Surgery Safer™



702 Prominence Drive
New Kensington, PA 15068,
United States

xodusmedical.com
info@xodusmedical.com
Tel: 1+724-337-5500

FDA REGISTERED
ISO 13485 CERTIFIED

LITIFUEB002 R10 10/3/2025