




# Jack Knife Lateral Positioning

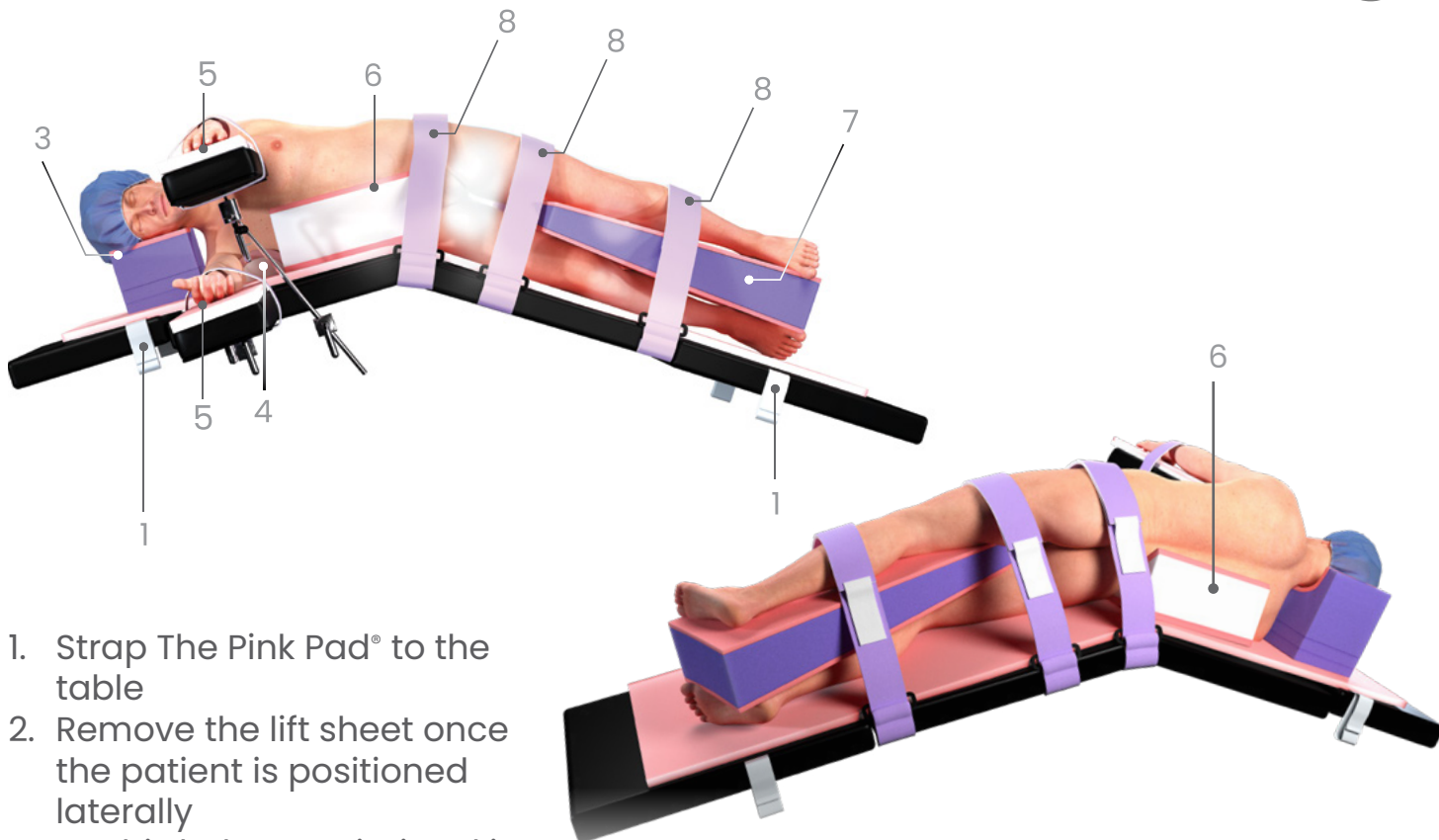

1. Strap The Pink Pad® to the table
2. Remove the lift sheet once the patient is positioned laterally
  - This helps maximize skin protection and allows wedges to lock to the Pink Pad
3. Place the #40471 Adjustable Lateral Head Positioner
4. Place the #40544 Pink Arc Positioner (axillary roll)
5. Position and protect arms (various options available)
6. Place the #40947 Low Profile Wedge, one in front and one behind the torso
7. Place the #40946 Tapered Intra-Leg Positioner
8. Secure the patient with body straps across the hip, middle of thigh, and middle of calf

