

# Pink Hip Kit - S + N

Instructions for Use

Postless Hip Positioning System

## Device Description

The Pink Hip Kit S+N is designed to be used with the Smith and Nephew Hip Positioning System.

The Pink Hip Kit S+N is single-use and discarded following completion of the surgical procedure.

The Pink Hip Kit S+N (Figure 1) consists of the following components:

- A. Head Rest (1)
- B. Pink Hip Pad (1)
- C. Body Strap (1)
- D. Lift Sheet (1)
- E. Armboard Straps (1pr)
- F. Armboard Pads (1pr)

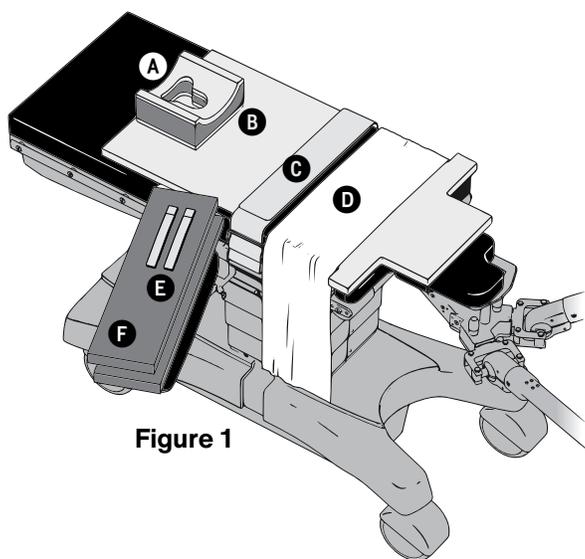


Figure 1

## Intended Users

The Pink Hip Kit S+N is intended to be used by healthcare professionals in accordance with these instructions for use. The use environment is a professional healthcare facility.

## Intended Use

The Pink Hip Kit S+N is designed to be used with the Hip Positioning System and prevents movement of the patient on the table during arthroscopic hip procedures.

## Indications for Use

The Pink Hip Kit S+N is indicated for use to position patients during arthroscopic hip procedures.

## Target Patient Groups

The patient population is comprised of patients for whom a hip arthroscopic procedure is determined to be necessary by the operating surgeon.

## Contraindications

- None

## Warnings

- It is the healthcare provider's responsibility to be familiar with product IFUs as well as training of relevant healthcare personnel.
- Product usage outside of indications or intended use is prohibited. Modifications to product may alter expected outcomes.
- During all handling and storage of The Pink Hip Kit S+N, assure that the Pink Hip Pad is flat. Do not roll or fold
- Make sure that the patient's skin is in direct contact with The Pink Hip Pad throughout the procedure.


## Precautions

- Inspect product packaging prior to opening. If product packaging is damaged or punctured, **DO NOT USE**.
- Inspect product and contents upon removal from packaging. If product is damaged or defective, **DO NOT USE** and immediately contact the manufacturer for support.
- Use caution when positioning surgical patients. Do not drag patients across the surface of the Pink Hip Pad, doing so may result in product damage and loss of function.

1. Refer to the Smith and Nephew Hip Positioning System Instructions For Use for system setup information.
2. Place the Pink Hip Pad on the OR table so that the alignment tabs line up with the Smith and Nephew Supine Table Attachment. (Figure 2) The text "This Side Up" should be facing upwards on the white hook & loop straps located on the Pink Hip Pad.

**Figure 2**


Align the tabs of the Pink Hip Pad with those of the Table Attachment.

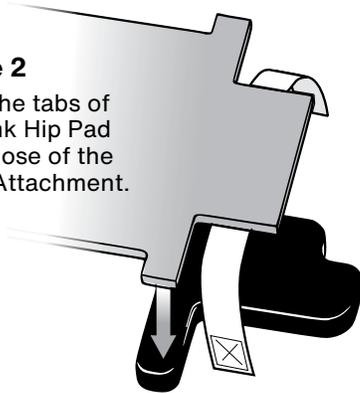

**CAUTION:** Make sure that The Pink Hip Pad makes direct contact with the surface of the OR table pad.

3. Attach all white hook & loop straps to the OR table rails by placing the loop portion of the tab down between the table and the accessory rail, then upward over the outside of the rail and affixing the hook portion of the hook & loop closure over the loop portion. (Figure 3)


**Figure 3**

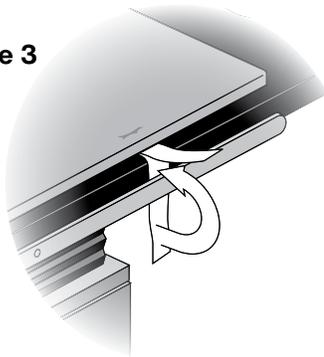

4. Affix the wide purple pad strap under the surgical table attachment by first placing the portion of the strap without the hook and loops underneath the table attachment and overlapping it with the remaining end of the strap, securing the hook & loop to ensure a snug fit. (Figure 4)
5. Place the Head rest on the Pink Hip Pad near the edge furthest from the hip attachment in a location sufficient for patient head placement. (Figure 1)
6. Center the Lift Sheet over the pad just above the alignment tabs. (Figure 1)

**CAUTION:** The lift sheet should cover only the portion of the pad that will address the small of the patient's back (between the scapulae and sacrum).


**Figure 4** View from under Table Attachment

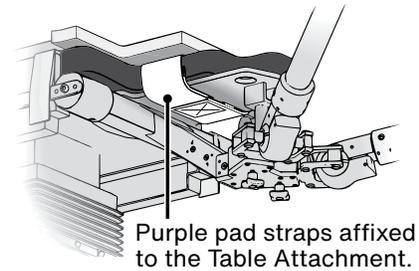

Purple pad straps affixed to the Table Attachment.

7. Secure both portions of the two-part body strap to the accessory rails:
  - Identify each portion of the strap – Part A has a small hook & loop square and Part B has a long hook & loop tab. (Figure 5)
  - Beginning with Part A, orient the strap so that the small hook and loop square is facing outward, away from the table pad. Draw the end of the belt up between the table and the accessory rail and slide the plastic buckle down over the end of the strap. Feed the strap through the buckle until it is snug to the accessory rail and lay the free end over the table.
  - Repeat the procedure with Part B of the strap on the other side of the table, ensuring that the hook side of the long hook & loop tab is facing inward toward the table pad.
  - Join straps together with the long hook & loop tab of Part B on top of Part

**Figure 5**

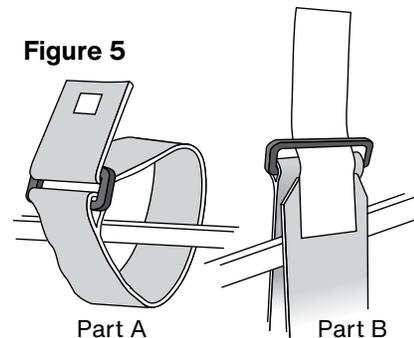


8. Transfer the patient to the Pink Hip Pad. Position the patient as follows:
  - The patient's anterior superior iliac spine (ASIS) should be positioned directly on the alignment tabs. (Figure 6)
  - Approximately six inches of the Pink Hip Pad will extend beyond the patient's gluteal crease.
  - Ensure that the shoulders do not extend past the top edge of the Pink Hip Pad, to maximize skin contact.
  - The skin of the scapular region and sacrum must contact the surface of the Pink Hip Pad.
  - If needed, utilize the lift sheet to carefully LIFT the patient up off of the pad to reposition. Do not drag the patient on the pad. Make sure the Pink Hip Pad remains flat at all times.

9. Place one purple Armboard Pad on the arm board of the OR table on the opposite side of the operative hip.
10. Place the patient's arm on the purple Armboard Pad with the palm up and secure it with an Armboard Strap.
11. Lay the other arm across the patient's chest, ensuring that the hand is in a natural, anatomically-correct position.
12. Place the remaining purple Armboard Pad between the patient's arm and chest, parallel with the patient's forearm. Center the purple Armboard Pad under the patient's hand and fold it back over the hand, sandwiching the patient's hand within the purple foam of the Armboard Pad.
13. Affix the purple two-part Body Strap around the patient, over the patient's elbow, arm and purple Armboard Pad, securing it with the hook & loop closure.


See Completed setup illustration for reference. (Figure 7)

**Figure 7**

Completed setup with properly positioned patient.

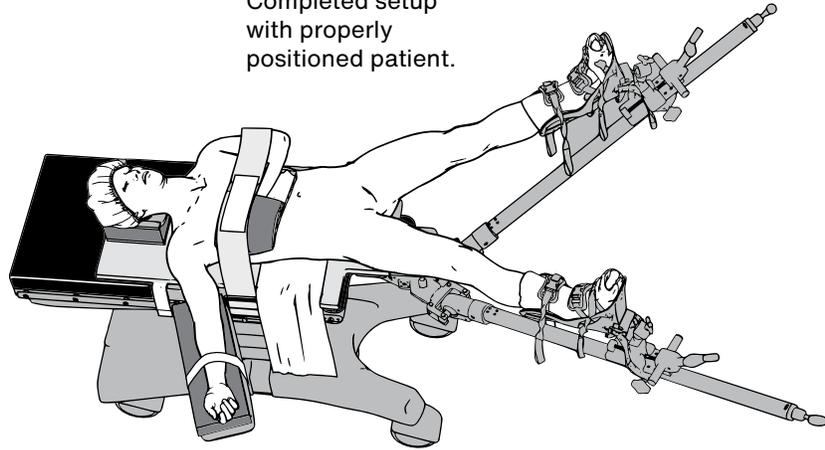

**Figure 6**

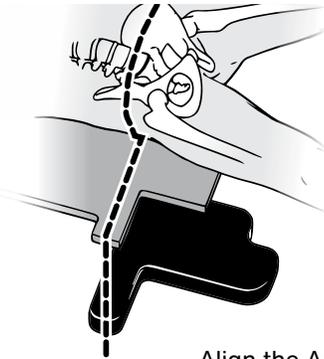

Align the ASIS with the tabs of The Pink Hip Pad and the Table Attachment.

\*Exclusively distributed by S+N

## Symbol Glossary

	Medical Device
	Caution
	Manufacturer
	Date of Manufacture
	Batch Code

	Do not use if package is damaged
	Do not re-use
	No Latex
	Quantity
	Catalog number

 <small>xodusmedical.com/efu</small>	Consult electronic instructions for use
	Non-Sterile
	Authorized representative in the European Community
	European Conformity



702 Prominence Drive | Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555

xodusmedical.com  
info@xodusmedical.com

FDA REGISTERED  
ISO 13485 CERTIFIED

LITIFUPP008 R6 1/18/2022