

DermaProxTM

Boot Liner - S+N

Instructions for Use

Device Description

The Smith & Nephew Dermaprox Boot Liners are designed to be used with the Hip Positioning System.

The Smith & Nephew Dermaprox Boot Liners are single-use and discarded following completion of the surgical procedure.

Intended Users

The Smith & Nephew Dermaprox Boot Liners are intended to be used by healthcare professionals in accordance with these instructions for use. The use environment is a professional healthcare facility.

Intended Use

The Smith & Nephew Dermaprox Boot Liners are designed to be used with the Hip Positioning System to help cushion and protect the foot when traction is applied and minimize foot and heel slip during hip arthroscopy procedures.

Indications for Use

The Smith and Nephew Dermaprox Boot Liners are indicated for use during hip arthroscopy procedures.

Target Patient Groups

The patient population is comprised of patients for whom a hip arthroscopic procedure is determined to be necessary by the operating surgeon.

Contraindications


- None

Warnings

- It is the healthcare provider's responsibility to be familiar with product IFUs as well as training of relevant healthcare personnel.
- Product usage outside of indications or intended use is prohibited. Modifications to product may alter expected outcomes.

Precautions

- Inspect product packaging prior to opening. If product packaging is damaged or punctured, **DO NOT USE**.
- Inspect product and contents upon removal from packaging. If product is damaged or defective, **DO NOT USE** and immediately contact the manufacturer for support.

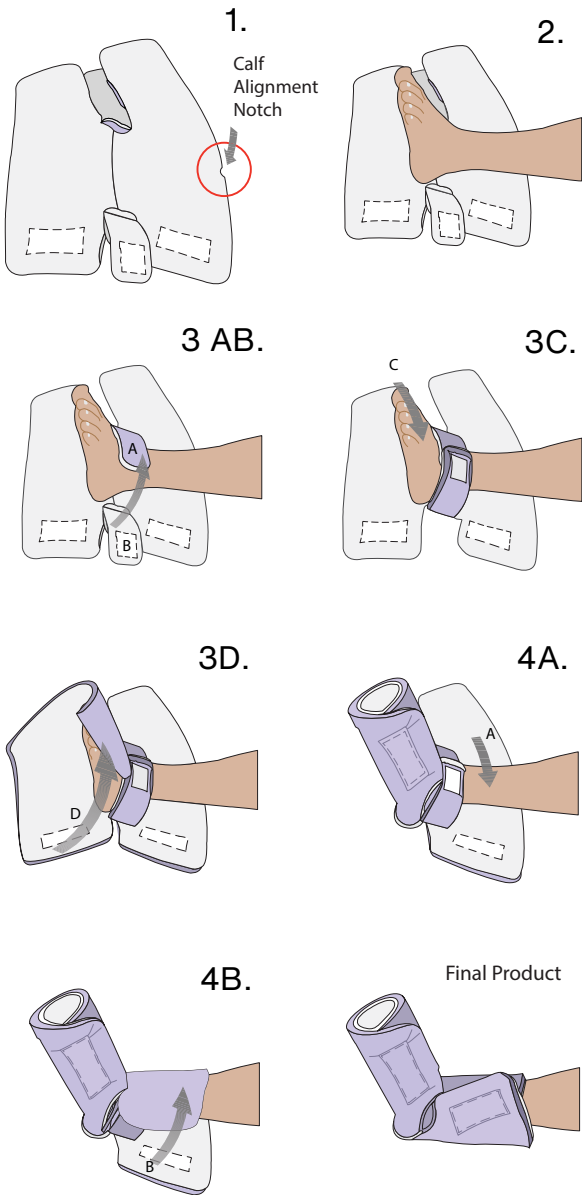

1. Unfold bootliner and have the white DermaProx™ layer facing upwards.
 2. Place patient's heel in the middle of the pad so that the notch aligns with patient's calf.

3A. Place strap A (non-hook & loop side) of middle strap on top of patient's ankle.

3B. Place strap B (hook & loop side) of middle strap on top of strap A so the patient's foot is wrapped. Be sure not to over-tighten.

3C. Wrap non-hook & loop side of the pad around the patient's toes.

3D. Wrap bottom half of pad around the rest of the patients toes and place the hook & loop on top of the non-hook & loop side of the pad so that the top of the patient's foot is wrapped. Be sure not to overtighten.

4A. Wrap non-velcro side of the pad around the patient's calf

4B. Wrap bottom half of pad around the rest of the patients calf and place the velcro on top of the non-velcro side of the pad so that the patient's foot is wrapped. Be sure not to overtighten.

**Exclusively distributed by S+N*

Symbol Glossary

	Medical Device
	Caution
	Manufacturer
	Date of Manufacture
	Batch Code

	Do not use if package is damaged
	Do not re-use
	No Latex
	Quantity
	Catalog number

	Consult electronic instructions for use
	Non-Sterile
	Authorized representative in the European Community
	European Conformity



702 Prominence Drive | Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
 Fax: 724.337.0555

xodusmedical.com
 info@xodusmedical.com

FDA REGISTERED
 ISO 13485 CERTIFIED