
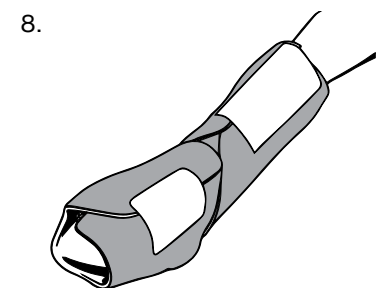
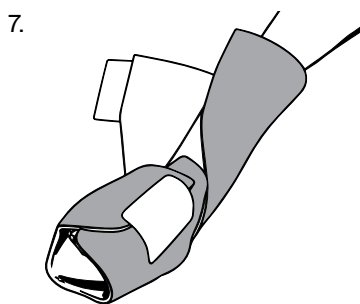
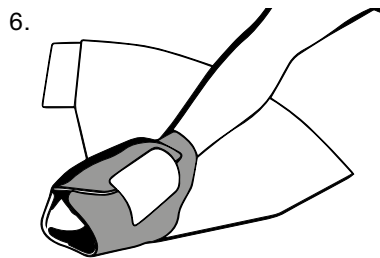
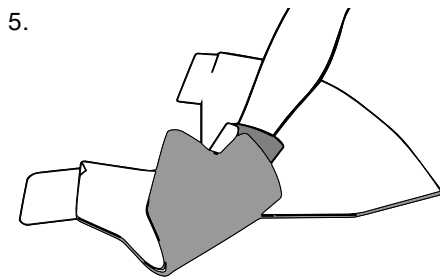
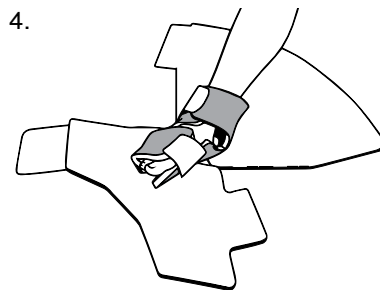
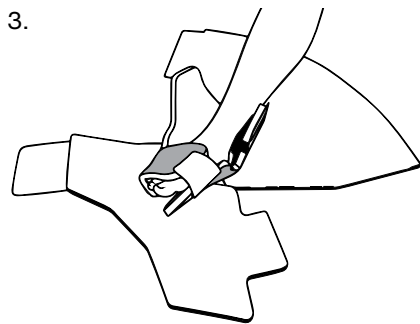
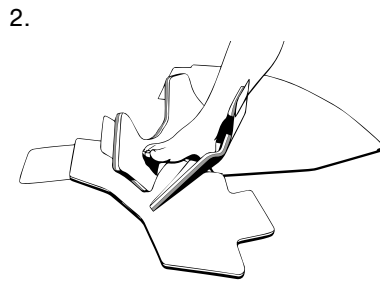


# INSTRUCTIONS FOR USE: DermaProx™ Boot Liner - Stryker Table

(Product #40536STR)

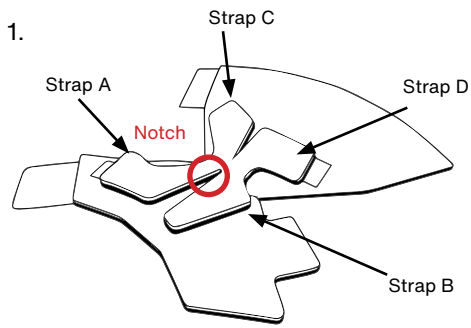

## Intended use

The DermaProx™ Boot Liners - Stryker are designed to be used with the Stryker Table to help cushion and protect the foot when traction is applied, and minimize foot and heel slip during hip procedures.

## Instructions

1. Unfold the boot liner and have the white DermaProx™ layer facing upwards.
2. Place the patient's heel in the middle of the pad so that the notch aligns with the patients heel.
3. Pull Strap A over the patient's instep, securing to Strap B with the hook strap.
4. Pull Strap C over the patient's tibia and secure with the hook strap to Strap D.
5. Wrap the right side of the foot cuff over the instep.
6. Wrap the left side of the foot cuff over the top of the right, and secure with hook strap.
7. Wrap the right side of the calf cuff across the calf.
8. Wrap the left side of the calf cuff across the calf and secure with hook strap.

## Warnings

- Product usage outside of indications or intended use is prohibited. Modifications to product may alter expected outcomes.
- Be sure not to over tighten.



702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555

xodusmedical.com  
info@xodusmedical.com

FDA REGISTERED  
ISO 13485 CERTIFIED

xodusmedical.com/eifu

LITIFUPP080 R0 8/19/2022