

INSTRUCTIONS FOR USE: One-Step™ Face Protector

(kit may not contain all products listed below)

1. Place the Headrest

Position the headrest as shown, with the top of the headrest closest to the anesthesia end of the table. The top of the headrest features the longer of two belts labeled forehead strap. (Figure 1)

2. Position the Patient

Position the intubated patient, centering the patient's head on the headrest. (Figure 1)

3. Place the One-Step Face Protector

Place the Face Protector over the patient's face as shown. Begin by lifting the velcro® tab on the chin guards of the Face Protector. Spread the chin portion at the dividing split, and pass the intubation tubing through the slit as shown. Place the Face Protector on the patient's face. Then replace the velcro closure. Insets of The Pink Pad® material on the inside of the Face Protector will rest on the patient's forehead, cheeks and chin. (Figure 2)

1. Affix the Head Straps

Affix the first portion of the purple forehead strap A located on the right of the patient's face to the velcro of the Face Protector. Secure the remaining half of the forehead strap B. The intubation tubing may be secured between the forehead straps. (Figure 3)

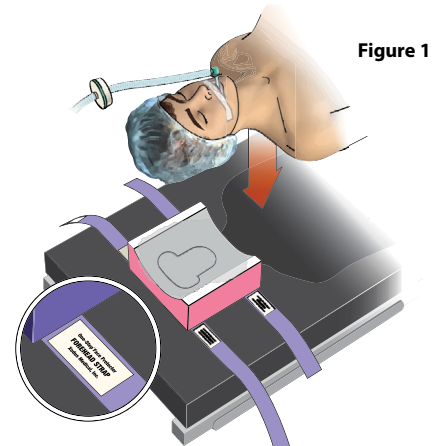


Figure 1

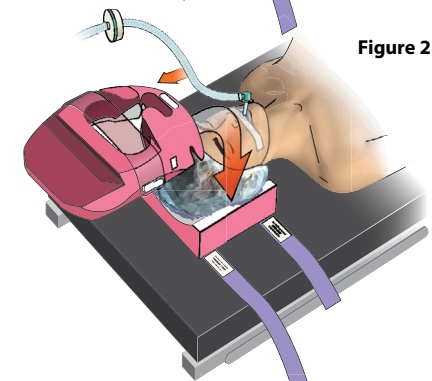


Figure 2

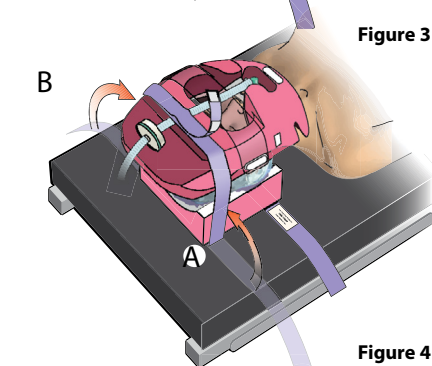


Figure 3


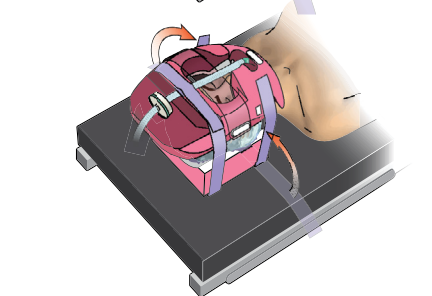


Figure 4



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555

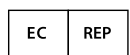

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



xodusmedical.com/eifu

xodusmedical.com
info@xodusmedical.com

FDA REGISTERED
ISO 13485 CERTIFIED

LITIFUPP001 R9 6/2/2021