



INSTRUCTIONS FOR USE: XODUS MEDICAL FACE PROTECTION

(kit may not contain all products listed below)


Placing On The Head

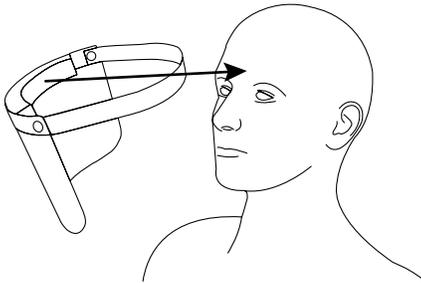

1.
Using one hand grasping the shield where the strap meets the shield, place the shield foam front against your forehead at a location above the eyebrows.

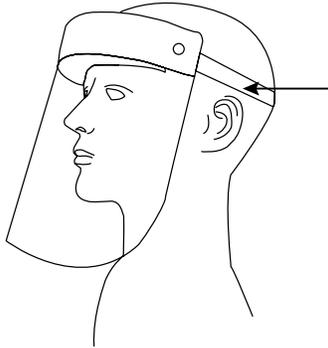


2.
While holding the foam front against your forehead, using your second hand, secure the elastic band around the back of your head so that the foam rests on your forehead.

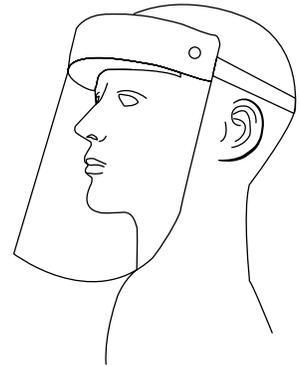


3.
Once shield is situated in a comfortable position, make sure the shield covers the front and sides of the head and that no areas are left uncovered.

Removing From The Head

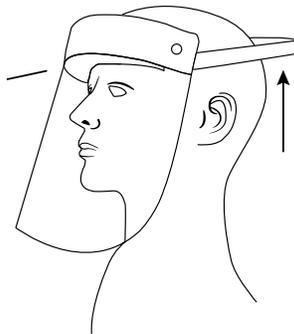

1.
Using one hand, grasp the front of the Face Protection Shield at the where the strap meets the shield.

2.
With your second hand, grasp the rear of the elastic band and lift the elastic band upward and forward above your head.

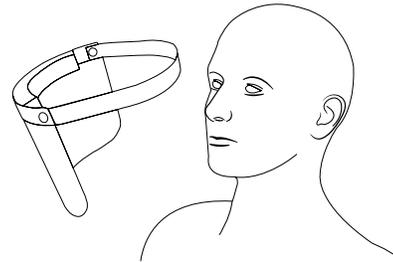

3.
Pull the shield away from your face.

4.
Dispose of the shield according to hospital protocol for the device.

When Using Your Face Mask:

1. Wear the face protection shield in addition to other personal protective equipment
2. Replace if wet
3. Avoid exposure to high heat
4. Handle gently to prevent drops and scratches
5. Do not share with others
6. Avoid exposure to high heat



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555



xodusmedical.com
info@xodusmedical.com

FDA REGISTERED
ISO 13485 CERTIFIED



xodusmedical.com/eifu

LITIFUFS001 R1 6/2/2021
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands