


INSTRUCTIONS FOR USE: HIP ARTHROSCOPY & FRACTURE KIT

1. Place The Pink Pad at the very edge of the table segment nearest the perineal post, positioning the white Velcro straps on the underside of The Pink Pad in contact with the table pad surface. The “This Side Up” tag should be facing upwards.
2. Attach the white Velcro straps to the surgical bed rails by looping under the rail and affixing the ends of the Velcro together.
3. Attach purple strap under the surgical table by first placing the longer end of the purple straps under the table, followed by overlapping the longer end of the strap with the shorter end of the strap. Make sure to press the white hook material of the shorter end of the strap firmly onto the longer end of the strap to secure in place.
4. Lay the lift sheet over the pad, centered between the Velcro straps as shown. The lift sheet should cover only the portion of the pad that will be addressing the small of the patient’s back – below the Scapula Region and above the Sacrum.
5. Place head positioner on top of Pink Pad in a location sufficient for the patient head placement.
6. Place Arm Board Pads onto the arm boards and affix the arm board straps temporarily.
7. Place Perineal Post Pad Cover over the perineal post, assuring it sufficiently all the way on the post.
8. Secure Ball Joint Covers over the ball joints of the table extensions by affixing the hook and loop of the cover.
9. Place arms on armboards with palms up, or per hospital positioning protocol.
10. Attach the body straps as follows:
 - a. Place the strap component with the small Velcro square around the table’s accessory rail and through strap buckle
 - b. Repeat the above step on the other side of the table with the remaining strap component with the Velcro hook side facing downward. Join the straps together.
11. Follow hospital protocol for intubation. Then properly position patient on pad.
 IMPORTANT-Position patient so that the shoulders do not extend past the edge of the pad.
 IMPORTANT-The skin of the Scapula Region and Sacrum must contact the pad surface
 Utilize the included lift sheet to carefully lift the patient up and off the pad to reposition as needed for safe and proper application of stirrups. Do not drag patient on pad. Make sure pad remains completely flat at all times.

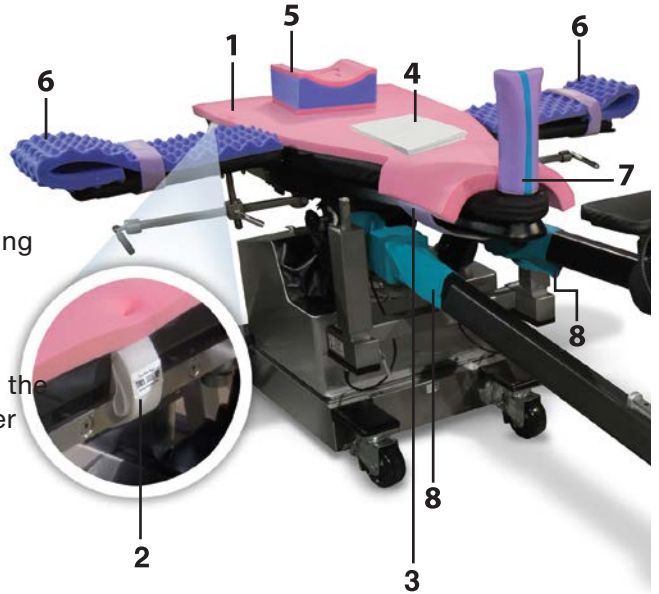

ATTENTION:

1. Before using The Pink Pad, ensure O.R. table pad is securely affixed to the O.R. table.
2. Be sure to follow your facility’s policies and guidelines for frequency of patient monitoring. Check for skin integrity and proper circulation. Product is to be used by a licensed medical professional only.
3. Handling & Storage: During ALL handling and storage, assure that the pad is flat. Do not roll or fold the pad.

LITIFUPP002 R7 5/26/2021

XODUS MEDICAL
Making Surgery Safer™



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500
Fax: 724.337.0555

xodusmedical.com
info@xodusmedical.com

FDA REGISTERED
ISO 13485 CERTIFIED

xodusmedical.com/eifu

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands