
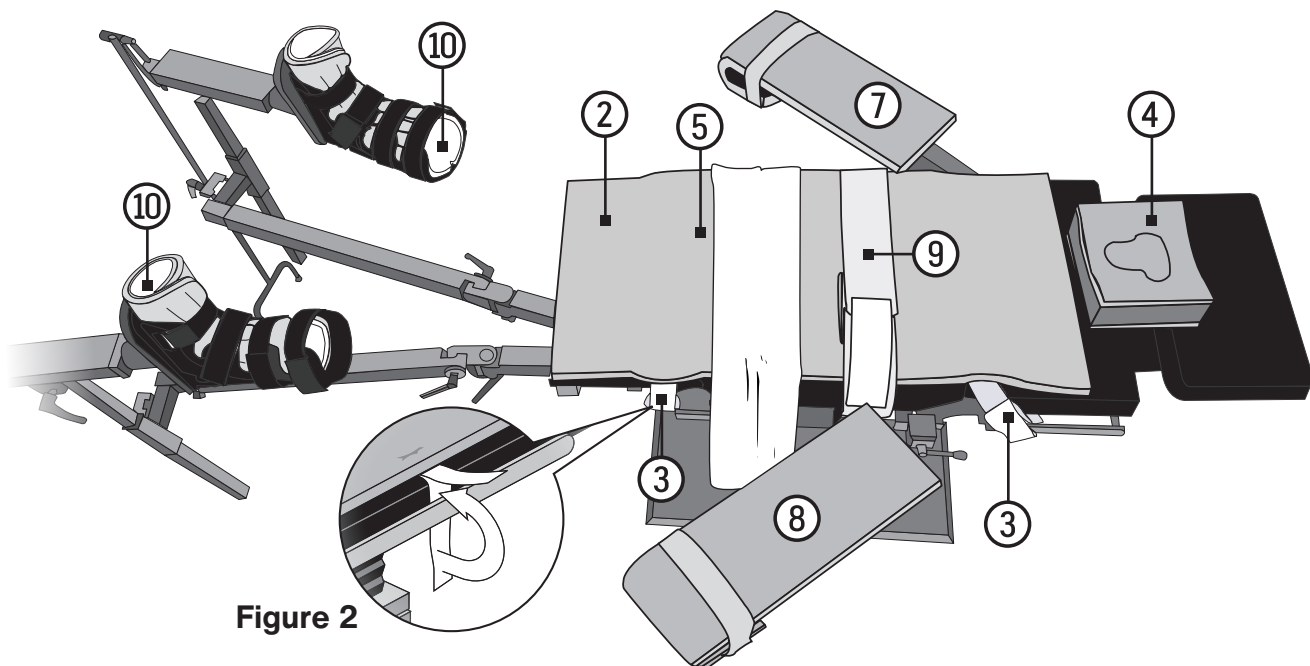


The Pink Hip Kit[®]

Postless Hip Positioning System


Nuovo BN II Table Instructions For Use

1. Before using The Pink Hip Kit, ensure O.R. table pad is securely affixed to the O.R. table.
2. Place The Pink Hip Pad in direct contact with the surface of the table mattress as indicated in Figure 1, aligning the bottom of the pad to the edge of the table covering the three post holes. The white Velcro straps of The Pink Hip Pad should be in contact with the table mattress with the “This Side Up” tags facing upward.
3. Attach each end (4) of both white Velcro straps to the surgical bed rails by looping under the rail and affixing the ends of the Velcro together as seen in Figure 2.
4. Place the head positioner above The Pink Hip Pad in a location sufficient for the patient head placement.
5. Lay the lift sheet over the pad, centered between the white straps. The lift sheet should cover only the portion of the pad that will be addressing the small of the patient’s back – below the Scapula Region and above the Sacrum. Begin IV or intubation according to hospital protocol.
6. Position the patient on The Pink Hip Pad with the lift sheet provided. Utilize the lift sheet to carefully lift the patient up off the pad to reposition as needed for safe and proper application. Do not drag patient on pad and make sure the pad remains flat all times.
7. Place one Arm Board Pad under the patient’s arm that is the opposite side of the operative hip. Place arm on arm board with palms up, or per hospital positioning protocol. See Figure 3 on reverse side.

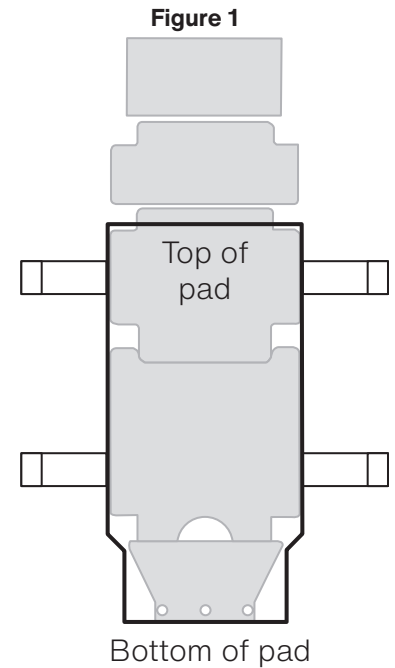

8. Place the other arm across patient's chest with the other arm board pad or on to the attached limb holder so that arm board pad cushions the patient's arm. Wrap the arm as indicated in Figure 3, and with accordance to hospital protocol.
9. Place purple body strap across patient's arm and attach the body straps as follows:
 - A. Place the strap component with the small Velcro square around the table's accessory rail and through strap buckle
 - B. Repeat the above step on the other side of the table with the remaining strap component with the Velcro's hook side facing downward. Join the straps together so that it goes over the patient's arm and chest.


Figure 3

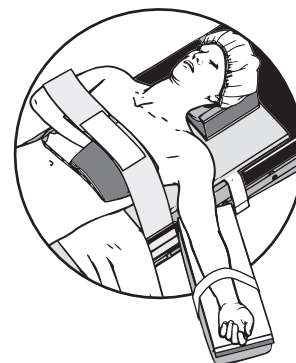


IF KIT INCLUDE BOOTLINERS, PLEASE FOLLOW THE ADDITIONAL STEPS BELOW

10. Place patient's feet in boot liners then wrap patient's feet according to hospital protocol. Place the non-operative leg into the attached foot stirrup. Tighten down according to hospital policy.
11. Place the patient's wrapped operative foot into foot stirrup, then tighten down according to hospital policy, and then begin postless hip distraction process.

Figure 4

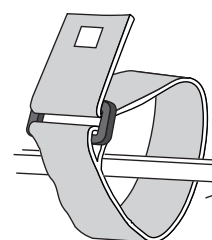

Part A

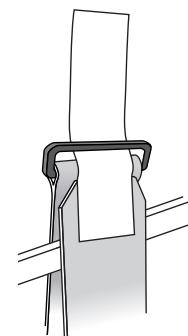

Part B

IMPORTANT:

- A. Position the patient's gluteal crease in alignment with the distal end of the pad.
- B. If the patient presents an anatomical challenge during attempted distraction, the use of Air Arthrogram could ease the distraction process.
- C. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Check for skin integrity and proper circulation. Product is to be used by a licensed medical professional only.



702 Prominence Drive
Pittsburgh, PA 15068 U.S.A. Tel: 724.337.5500
Fax: 724.337.0555



xodusmedical.com
info@xodusmedical.com



FDA REGISTERED
ISO 13485 CERTIFIED



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands