


# INSTRUCTIONS FOR USE: Reusable Knee and Body Strap

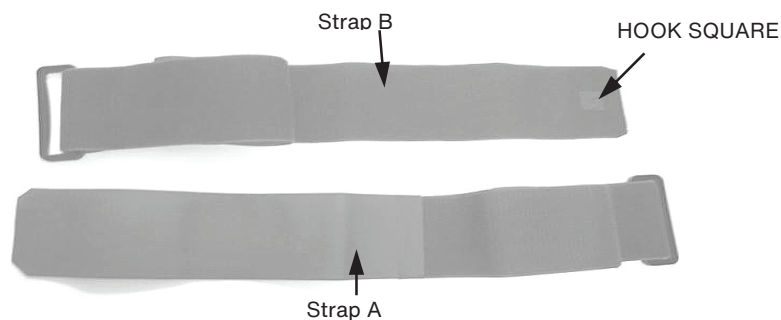


## ATTACHING STRAPS TO BED/TABLE:




### Step 1

Slide end of strap A around bed/table frame then through plastic clasp. Make sure hook is face down on end. Repeat with strap B around other side of bed/table making sure hook square is face up. Smooth side should always face toward.

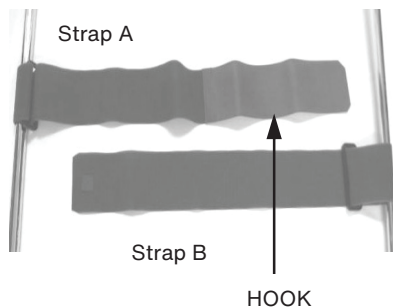

### Step 2


Belts should look like this when properly attached to table/bed. Hook side is facedown on strap A.

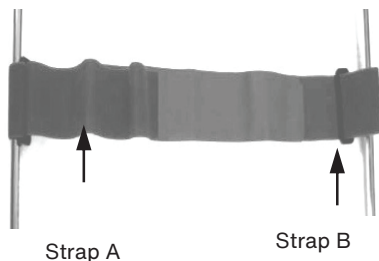

### Step 3


Place the end of strap A across strap B to fasten in place.

## ATTACHING STRAPS TO EACH OTHER:

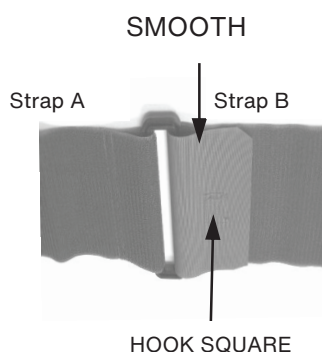


### Step 1

Slide the end of strap B through the plastic clasp at the end of strap A and fasten. The smooth side should always face toward the patient.

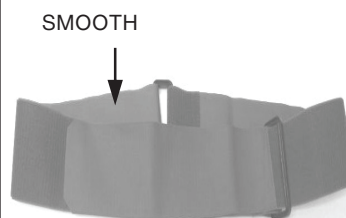

### Step 2

Place the strap around the table/bed and up around the patient. Slide the end of strap A through the plastic clasp of strap B and across to fasten.

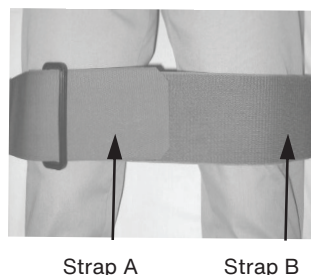

This is how the straps should look when fastened across the patients legs.

LITIFURS001 R7 5/21/2021



702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555



xodusmedical.com  
info@xodusmedical.com



FDA REGISTERED  
ISO 13485 CERTIFIED



xodusmedical.com/eifu

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands